

English

# OWNER'S MANUAL

INTRAORAL DIGITAL  
RADIOGRAPHIC SYSTEM

new|D|Λ



## PRESENTATION OF MANUAL

**GMDN:** 44905 - Dental digital imaging sensor, intraoral

**Model:** New IDA

**Brand:** Dabi Atlante

**Technical Responsible:** Ricardo José Ravaneli

**CREA-SP:** 5060714523

**ANVISA Register:** 10101139025



Alliage S/A Industrias Médico Odontológico  
Rodovia Abrão Assed, Km 53 - CEP 14097-500  
Ribeirão Preto - SP - Brazil  
Telephone: +55 (16) 3512-1212  
CNPJ: 55.979.736/0001-45



CINTERQUAL – Soluções de Comércio  
Internacional, Lda.  
Rua Fran Pacheco N°220 2° And  
2900-374 Setúbal  
Portugal



77000000592 - Rev.: 13 – June/2019

Document originally written in Portuguese.

## **TRADEMARKS**

All terms stated in this manual that are from known trademarks, commercial trademarks or service marks were properly labeled as such. Other products, services or terms that are mentioned in this manual can be trademarks, commercial trademarks or service marks of their respective owners. The Alliage S/A makes no claim in relation to these trademarks. The use of a term in this manual must not be considered as influential over any trademark, commercial trademark or service mark.

New IDA is a commercial trademark from Alliage S/A.

Microsoft, Windows and .NET Framework are commercial trademarks of Microsoft Corporation in the United States and other countries.

GNU LESSER is a trademark from Free Software Foundation, Inc.

Copyright © 2017 Alliage S/A. All Rights Reserved.

The performance characteristics presented in this manual are only for reference and mustn't be considered as secured specifications.

## TABLE OF CONTENTS

<b>DEAR CUSTOMER</b> .....	<b>6</b>
<b>USE INDICATION</b> .....	<b>6</b>
<b>CONTRAINDICATIONS</b> .....	<b>6</b>
<b>SYMBOLOLOGY</b> .....	<b>7</b>
<b>WARNINGS, CAUTIONS, AND RECOMMENDATIONS</b> .....	<b>8</b>
<i>DURING TRANSPORTATION AND STORAGE</i> .....	8
<i>BEFORE USING THE EQUIPMENT</i> .....	9
<i>DURING THE INSTALLATION OF THE EQUIPMENT</i> .....	9
<i>DURING THE EQUIPMENT USE</i> .....	9
<i>Prevention against cross contamination</i> .....	10
<i>AFTER USE / OPERATION OF THE EQUIPMENT</i> .....	10
<i>PRECAUTIONS IN CASE OF CHANGE IN THE PERFORMANCE OF THE EQUIPMENT</i> .....	10
<i>PRECAUTIONS FOR THE REDUCTION OF ENVIRONMENTAL IMPACT</i> .....	11
<i>PRECAUTIONS IN CASE OF DESTRUCTION OF THE EQUIPMENT</i> .....	11
<b>SYSTEM DESCRIPTION</b> .....	<b>12</b>
<i>SYSTEM STRUCTURE</i> .....	12
<i>INTRAORAL SENSOR</i> .....	13
<i>INTRAORAL SENSOR HOLDER</i> .....	13
<i>COMPUTER SYSTEM</i> .....	13
<i>Software requirements</i> .....	13
<i>Hardware requirements</i> .....	14
<i>Electrical requirements</i> .....	14
<i>Installation site</i> .....	14
<i>SOFTWARE</i> .....	15
<i>Software installation</i> .....	15
<i>Main screen</i> .....	20
<i>Setting screen</i> .....	24
<i>Odontogram</i> .....	26
<i>CONSUMABLES</i> .....	31
<i>LABELS</i> .....	32
<i>Identification</i> .....	32
<i>Safety</i> .....	32
<i>Refer to instruction manual</i> .....	32
<i>INTRAORAL RADIOGRAPHY EQUIPMENT</i> .....	33
<i>SPECIFICATIONS AND TECHNICAL CHARACTERISTICS</i> .....	34
<i>General information</i> .....	34
<i>Characteristics of the sensor</i> .....	34
<i>Environmental conditions for transportation and storage</i> .....	35
<i>Conditions for installation and operation of the equipment</i> .....	35
<b>CERTIFICATIONS AND STANDARDS</b> .....	<b>36</b>

EMC / Security .....	36
Usability .....	36
Software .....	36
Quality.....	36
<b>SYSTEM OPERATION .....</b>	<b>37</b>
<i>PREPARATION FOR USE</i> .....	37
<i>ACQUISITION OF RADIOGRAPHIC IMAGES</i> .....	38
<i>Create a new registration of patient</i> .....	39
<i>Opening a patient registration</i> .....	40
<i>Changing a patient registration</i> .....	40
<i>Sensor status</i> .....	40
<i>Process of image capturing</i> .....	41
<i>Images of the patient's registration</i> .....	42
<i>Metric Calibration</i> .....	44
<i>Precision of the measurement</i> .....	47
<b>RECOMMENDATION FOR EXAMS .....</b>	<b>48</b>
<b>CLEANING AND DISINFECTION.....</b>	<b>51</b>
<b>MAINTENANCE .....</b>	<b>52</b>
<i>PERIODIC INSPECTION</i> .....	52
<i>PREVENTIVE MAINTENANCE</i> .....	52
<i>CORRECTIVE MAINTENANCE</i> .....	52
<i>ERROR DIAGNOSIS</i> .....	54
<i>CABLE REPLACEMENT</i> .....	55
<i>AUTHORIZED SERVICE NETWORK</i> .....	58
<i>WARRANTY TERM</i> .....	58
<i>Warranty period</i> .....	59
<b>ELECTROMAGNETIC COMPATIBILITY .....</b>	<b>60</b>
<i>GUIDANCE AND DECLARATION FOR ELECTROMAGNETIC IMMUNITY</i> .....	61

## **EQUIPMENT IDENTIFICATION**

### **DEAR CUSTOMER**

Congratulations for your excellent choice. When buying equipment with the "DABI ATLANTE" quality, you can be sure you purchased products with technology that is compatible with the best in the world from its class.

This manual offers you a general presentation of your equipment, describing important details that can guide you in its correct use, as well as in the solution of small issues that can eventually occur. There is no need of any additional training besides its own reading.

This manual must be read on the whole and preserved to future consultations.

### **USE INDICATION**

The Intraoral dental digital imaging sensor New IDA is aimed at the acquisition of intra-oral medical images from teeth, jaw and oral structure, only for exclusive dental use and must be handled by trained and qualified health professionals, observing the instructions included in this manual.

### **CONTRAINDICATIONS**

The intraoral dental digital imaging sensor New IDA is contraindicated to patients with temporomandibular dysfunction that limited mouth opening and who have anatomical characteristics that prevent their use as a low palate.







The protective plastic covers contain polyvinyl chloride (PVC) and should not be use by people with allergies to polyvinyl chloride (PVC) or substances used in the manufacturing process of the same such as bisphenol A, adipic polyester, propylene glycol compound and ethylhexyl maleate

## SYMBOLOLOGY

Use the icons above to identify the symbology of your equipment.

	<b>Fragile, handle with care</b>		<b>Applied parts BF type</b>
	<b>Keep dry</b>		<b>Equipment Class II</b>
	<b>This side up</b>		<b>Attention</b>
	<b>Keep away from sunlight</b>		<b>General warning</b>
	<b>Maximum stacking</b>		<b>Mandatory action</b>
	<b>Temperature limit</b>		<b>Follow the instructions for operation</b>
	<b>Humidity limit</b>		<b>USB connection</b>
	<b>Recyclable</b>		<b>Catalog number</b>
	<b>Do not re-use</b>		<b>Serial number</b>
	Indicates that the product must be taken to a special waste collection site at the end of its lifespan. Applies to the device and to the accessories.		<b>Part Number</b>
<b>IP00</b>	This product is not protected against harmful water infiltrations and particulate matter.		<b>Authorized representative in the European Community</b>
	Indicates that the product has passed an evaluation, and that standards or regulations developed for the product category have been observed in its design / manufacture / placing on the market.		<b>Manufacture date</b>
	Indicates that the equipment is in compliance with Directive 2011/65 / EU on restriction of the use of certain hazardous substances in electrical and electronic equipment.		<b>Manufacturer</b>
	Indicates that the equipment is in compliance with the Guideline 93/42 / CEE of European Union about Medical Devices (MDD)		

## WARNINGS, CAUTIONS, AND RECOMMENDATIONS

	Read and understand the instructions contained in these instructions for use before install or operate this equipment.
	The operation of this device may be dangerous for patient and operator unless safe exposure factors and operating instructions are observed. Personal protective equipment must be used with this device.
	Use the equipment only if in perfect conditions and protect yourself, patients and third parties against possible dangers.
	This equipment must be installed and operated by personal acquainted to the necessary precautions to avoid excessive exposure to primary and secondary radiation.
	<p>The New IDA sensor has three different interactions with the user. They are:</p> <p>Identification label: Placed in the lower part of the product package;</p> <p>Security symbology: Placed in the upper part of the sensor on its identification;</p> <p>- Image acquisition software: Found installed with a computer where the sensor is connected.</p>
	<p>This equipment meets the requirements of the directive RoHS (<i>Restriction of Certain Hazardous Substances</i>) (Directive 2011/65/UE) and the Directive of Medical Devices (Directive 93/42/EEC). Please note that the compliance with the directives RoHS is ensured by the exclusive responsibility of the manufacturer.</p> <p>Non-authorized modification of the product or of the configuration invalidate the CE marking</p>

### ***DURING TRANSPORTATION AND STORAGE***

The equipment must be shall be carried and stored according to the following:

- Handle with care to avoid falls and impacts.
- The arrows on the package must be pointing upwards.
- Don't stack items above the quantity indicated on the packaging
- Protect against sunlight, humidity, water and dust.
- Observe the limit of temperature, pressure and relative humidity

### ***BEFORE USING THE EQUIPMENT***

To help and ensure a proper hygiene and to protect against infectious diseases, before first use, the equipment must be cleaned and disinfected according to the instructions enclosed in this manual.

### ***DURING THE INSTALLATION OF THE EQUIPMENT***


- Put the equipment in a place where it won't be in contact with humidity or water.
- Install the unity in a place where it won't be damaged by pressure, temperature, humidity, direct sunlight, dust, salt or corrosive products.
- The equipment must not be subjected to excessive vibration or shock (including during transportation and handle).
- This equipment was not designed for use in presence of vapors from flammable anesthetics mixtures or nitrous oxide.
- The recommendations of this manual must be followed regarding the EMC. Communication equipment and sources that generate RF can affect the performance of the equipment.

### ***DURING THE EQUIPMENT USE***



**Proper hygienic measures must be taken to avoid cross contamination among the patient, users and other people.**

- The equipment must be operated only by qualified healthy professionals including safeguard knowledge against excessive exposure to radiation.
- The equipment was manufactured to resist continuous and intermittent operation; therefore, follow the cycles described in the Owner's Manual.
- X-Ray exposure can cause damage to human body cells. It is recommended that no one stays in the radiographic examination room unless the containment of the patient is necessary. In this case, this person must be properly protected against ray emissions.
- Operators should follow all radiation protection guidelines for patients who are pregnant or suspected of pregnancy, including the use of shielding of the abdomen and thyroid.
- The equipment was projected according to the electromagnetic compatibility standards, but in very extreme conditions, it can cause interference with other equipment. Don't use this equipment along with other devices that are sensitive to interference or with devices that create high electromagnetic disturbances.

	<p><b>The manufacturer assumes NO liability:</b></p> <ul style="list-style-type: none"> <li>• <b>If the equipment is used for other purposes that are not the ones for what it was originally designed.</b></li> <li>• <b>For damages caused to the equipment, operator and/or patient as result of incorrect installation and maintenance procedures that are not in compliance with the operation instructions that comes with the equipment.</b></li> <li>• <b>For inappropriate operation of the equipment.</b></li> </ul>
---	--

### ***Prevention against cross contamination***


- To help to prevent the cross contamination among the patients, it is mandatory the use of disposable protective plastic covers.
- The covers must be properly disposed of for each new patient.
- The protective plastic cover must cover the sensor and at least 10 cm of the USB cable.
- Don't ever use damaged protective plastic covers.

### ***AFTER USE / OPERATION OF THE EQUIPMENT***

- Turn off the equipment when it is not being used for an extended period.
- The protective plastic covers must be disposed for each new patient to avoid the transmission of infectious agents that can cause serious conditions.
- Perform the cleaning and disinfection according to the instructions contained in this manual.
- Don't modify any part of the equipment.

### ***PRECAUTIONS IN CASE OF CHANGE IN THE PERFORMANCE OF THE EQUIPMENT***

If the equipment presents any abnormality, verify id the problem is related to any item listed in the topic: "Error Diagnosis" of this user manual. If it is not possible to solve the problem, turn off the equipment, disconnect the cables.

	<p><b>In the event of damage or defect do not use the equipment and contact an Authorized Service Alliage.</b></p>
---	--

### **PRECAUTIONS FOR THE REDUCTION OF ENVIRONMENTAL IMPACT**

Alliage S/A aims to reach an environmental politics to promote the supply of medical and dental products environmental conscious that continuously minimize the environmental impact and that are more friendly to the environment and to the human health.

To keep a minimum impact on the environment, observe the following recommendations:

- After installation, forward the recyclable materials for the recycling process.
- During the life cycle of the equipment, turn it off when it is not being used.
- To prevent environmental contamination, the disposal of the protective plastic covers and other consumables must be according to the normal procedure of the dental office for biomedical waste.

The biomedical waste comprises nonhigh materials susceptible to cause diseases or suspect for housing pathogenic organisms that must be stored in a yellow bag properly labeled with the symbol of biological hazard, stored in a container resistant to perforation, tight and even recollection and incineration.

	<p><b>The packaging<sup>1</sup> of the New IDA sensor is composed of cardboard and plastic, which are 100% recyclable materials.</b></p>	
---	--	---

<sup>1</sup> DIMENSIONS: 185 x 160 x 71.5 mm /WEIGHT: Approximately: 100g

### **PRECAUTIONS IN CASE OF DESTRUCTION OF THE EQUIPMENT**

To avoid environmental contamination or inappropriate use of the New IDA when it is destroyed, they must be disposed of (according to the current legislation) in an appropriate place, since the existent materials inside it can contaminate the environment.

To the European Economic Area (EEA), this product is subject to the Directive 2012/19/EU, as well as to the correspondent national laws. This directive requires that the product must be taken to a special waste collection site at the end of its lifespan. Applies to the device and to the accessories

Contact the retailer if the final disposition of the product is necessary.

	<p><b>This equipment must not be disposed of as domestic waste.</b></p>
---	---

## SYSTEM DESCRIPTION

### SYSTEM STRUCTURE



- 1 – Intraoral sensor
- 2 – Computer system<sup>1</sup> (USB interface)
- 3 – Installation flash drive (Software, Drivers & Manuals)
- 4 – Intraoral sensor holder
- 5 – Consumables (Plastic Protective Covers<sup>1</sup>)

<sup>1</sup>Don't accompany the product

## **INTRAORAL SENSOR**

The Intraoral dental digital imaging sensor New IDA uses a sensor that employs the associated technologies from CMOS (*Complementary Metal-Oxide-Semiconductor*), protective optical fiber and scintillator. This sensor was developed to obtain a high-quality x-ray image from the human arch and its structures. The acquisition process is made by positioning the sensor inside the mouth, behind the structure you want to perform the exam. The structure must be exposed to an x-ray dose using an external source. Once exposed, the sensor performs a conversion of the x-ray photons into a digital sign and transfers it to a computer through USB 2.0 connection (*Universal Serial Bus*).

## **INTRAORAL SENSOR HOLDER**

The intraoral sensor holder consists of a base where the sensor can be placed when it is not in use.

The intraoral sensor holder can be screwed close to the computer system using the two supplied screws or other mounting material.

Alternatively, it may be glued to a uniform surface using the supplied double-sided adhesive tape.

## **COMPUTER SYSTEM**

### **Software requirements**

For the correct performance of the New IDA software, some basic software described below is necessary:

**.Net Framework 4.5.1:** The NET Framework is essential to run several programs that were developed through the use of this technology from Microsoft. This framework offers a series of resources for the agile and robust development of applications. The 4.5.1 version requires the Windows Vista SP2 or superior and it's included in Windows 8.1 and in Windows Server 2012 R2.


**wkhtmltopdf 0.12.3.2:** It's an open-source command line tool (LGPLv3) to render HTML to PDF using the rendering mechanism from Qt WebKit. It works completely "headless" and it doesn't require an exhibition service or exhibition.

The required software for the performance of the New IDA software will be automatically installed, in shape of prerequisites, during the installation of the application, offering more speed and convenience to the user.

## Hardware requirements

The computer system must meet the following requirements.


<b>Operational System</b>	Windows 7 (x86 e x64) - Home, Starter, Pro Windows 8 (x86 e x64) - Pro Windows 8.1 (x86 e x64) - Pro Windows 10 (x86 e x64) - Home, Pro
<b>CPU</b>	Intel i3 3.0 Ghz or superior
<b>HDD</b>	500 GB
<b>RAM</b>	2GB DDR2 SDRAM
<b>Monitor</b>	1024x768
<b>USB connector</b>	2 x USB 2.0 – 500mA

	<p><b>The computer must be approved by CE and must be in conformity with the standard EC 60950-1:2005 + AMD1:2009+AMD2:2013 and the Low Voltage 2014/35/EU and the EMC 2014/30/EU Directives.</b></p> <p>Besides, to help ensure the ideal performance, make sure that all the installed programs are free from virus and were properly tested so they won't affect the image acquisition software after its installation.</p>
---	--


	<p><b>It is recommended the execution of regular backup in the files and patient data to avoid record loss in case of failure of the computing system.</b></p>
---	--

## Electrical requirements

The New IDA sensor doesn't require an external power supply since its power is supplied through the USB connection of the computer and it has approximate consumption of 300mA.

	<p><b>Don't use USB ports available in keyboards or in the front or lateral side of the computer. These connections are frequently unable to deliver sufficient power for the proper operation of the sensor.</b></p>
---	---

## Installation site

	<p><b>To meet the safety regulations, don't operate non-medical equipment like personal computers inside the patient area. Outside the patient area, it's acceptable the presence of non-medical equipment as long as it's used computer equipment approved and certified</b></p>
---	---

Place your computer and video monitor outside the patient area, ensuring that there's enough space for proper ventilation.

Preferably, use the monitor in 4:3, with full HD features and place the screen to avoid direct light or reflexes that can make difficult the visualization of the images.

Place the sensor next to the patient, with a minimum distance of 1.5 meter from the computer. Only the sensor and medical equipment can be inside the patient area.

## **SOFTWARE**

The dental image management software performs the interface with the Intraoral dental digital imaging sensor New IDA and the user, allowing the visualization, improvement, storage, sending, printing among other functions.

### **Software installation**

Insert the sensor in the USB port of the computer




Connect the flash drive that accompanies the New IDA in the USB port, open the device and click on the file **setup\_New\_IDA\_2\_0\_0\_X**




Select the desired language and click on **OK**




Read and understand the End User License Agreement and, if you accept, click on **I accept the terms of the Contract**. Click **Next**.

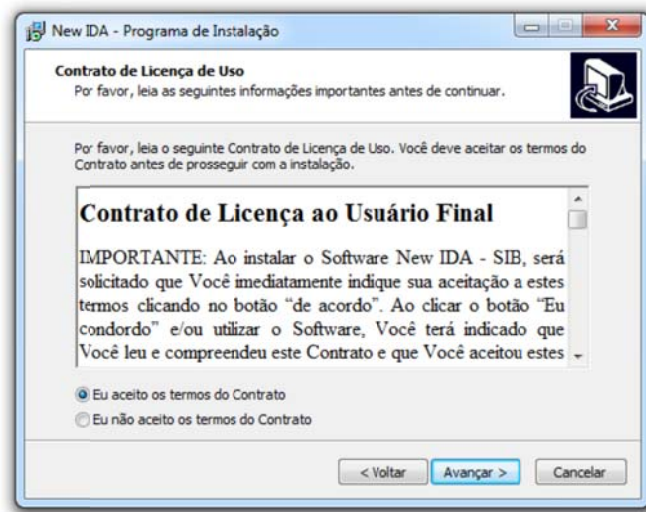


Select the dialog box if you want to create an icon on Desktop and click **Next**

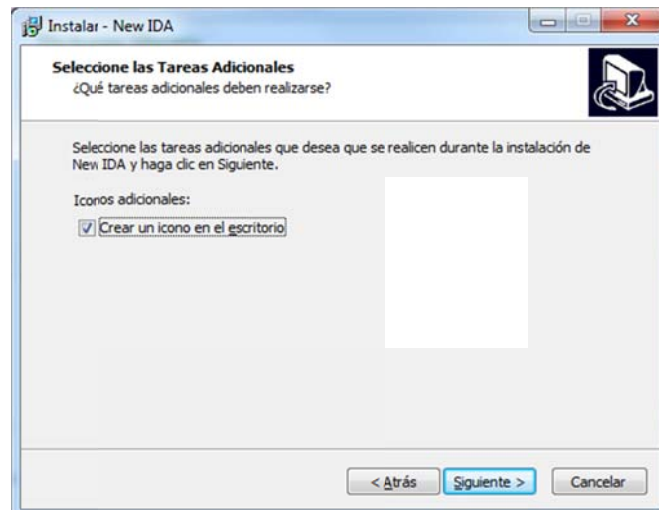


Click **Install** to begin the installation process.

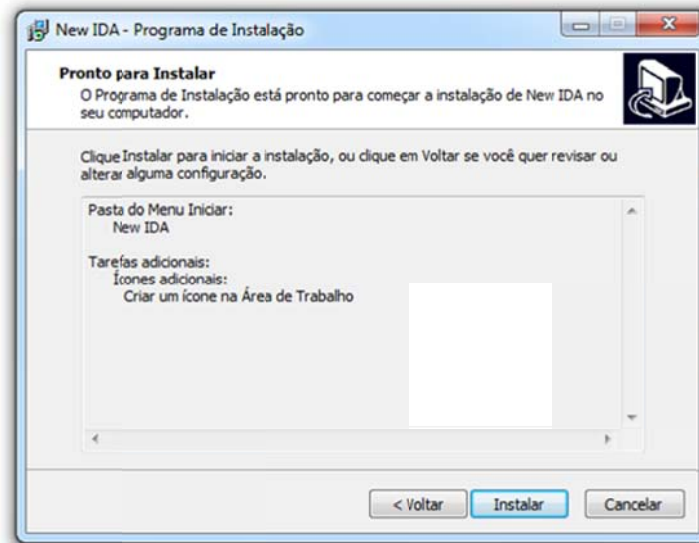


Setup will install .NET 4.5.1

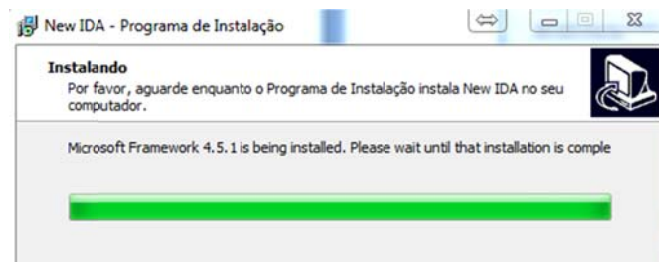


Setup will install the necessary tools to generate reports. Click "I Agree" to accept the terms of the software

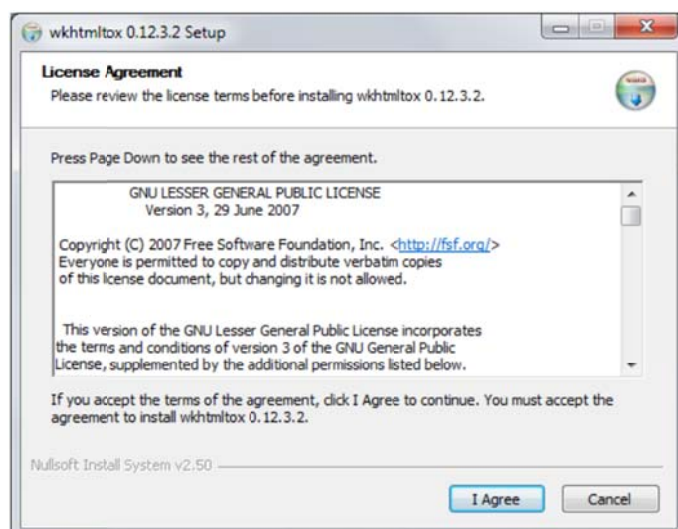

Click "Install"

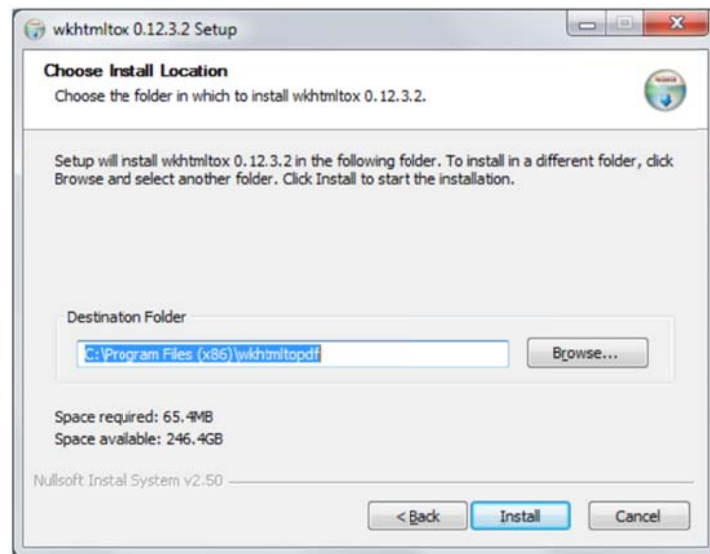

Wait for the end of installation and click **"Close"**

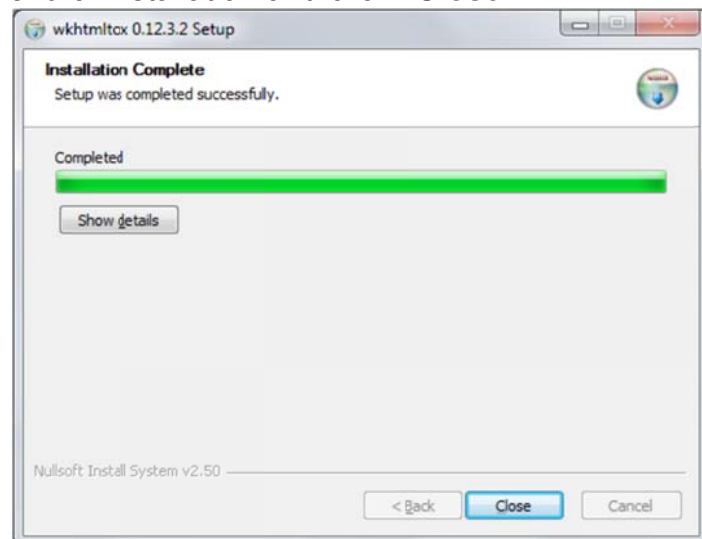


Setup will install the USB driver. Press **Install** to begin the installation process.



Click **Finish** to end installation.



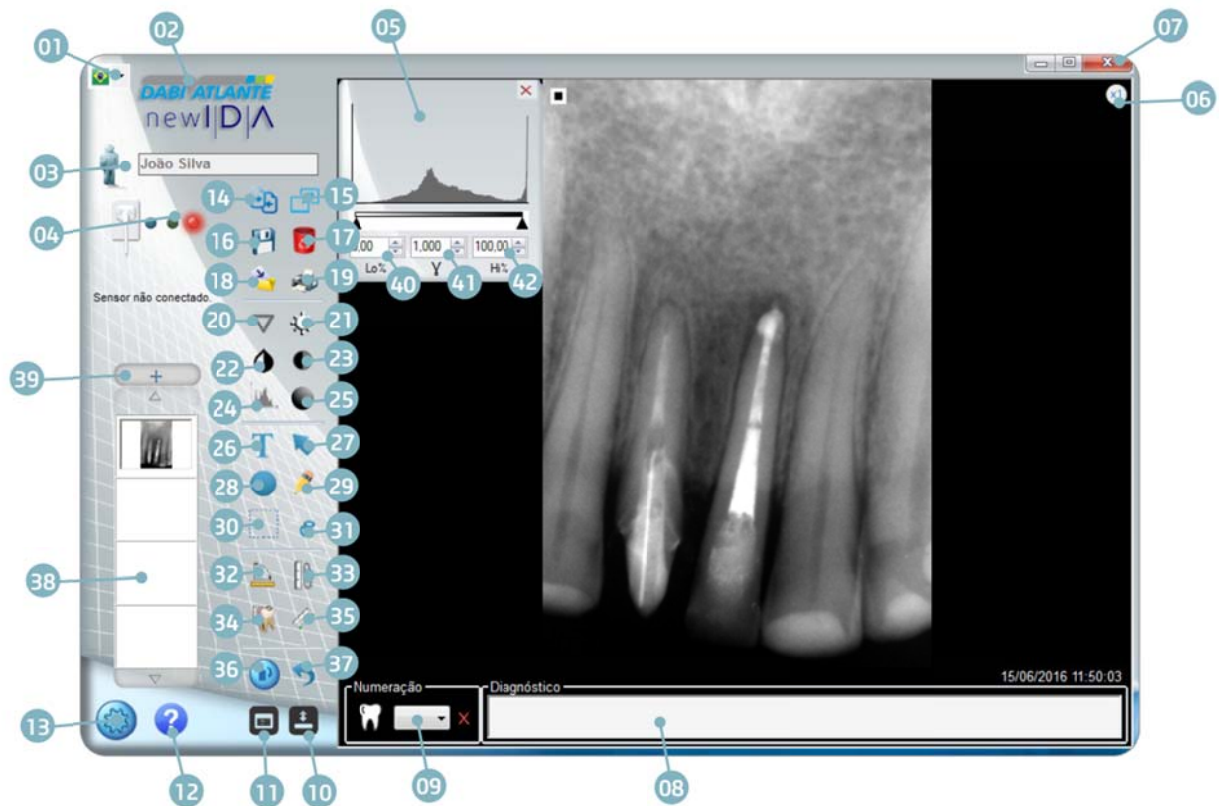













**Main screen**





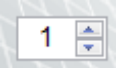












**01 - Language selection**  
**02 - Access to the manufacturer site**  
**03 - Patient folder**  
**04 - Sensor status**  
**05 - Histogram window**  
**06 - Fixed Zoom 1x or 2x**  
**07 - Exit**  
**08 - Number tagging and diagnosis**  
**09 - Tooth selection**  
**10 - Show / Hide toolbar**  
**11 - Odontogram**  
**12 - Help**  
**13 - Setting**  
**14 - Compare images in full screen**  
**15 - Full Screen**  
**16 - Save**  
**17 - Delete**  
**18 - Export image**  
**19 - Print**  
**20 - Sharpen**  
**21 - Brightness setting**


















**22 - Invert colors**  
**23 - Contrast setting**  
**24 - Histogram**  
**25 - Gamma setting**  
**26 - Insert Text**  
**27 - Insert Arrow**  
**28 - Insert circle**  
**29 - Insert drawing**  
**30 - Select**  
**31 - Cropping**  
**32 - Angular measurement**  
**33 - Calibrate**  
**34 - Multiple measurement**  
**35 - Single Measurement**  
**36 - Rotate**  
**37 - Undo**  
**38 - Open images**  
**39 - Opening the Patient's Folder**  
**40 - Histogram Adjustment - minimum**  
**41 - Gamma Adjustment**  
**42 - Histogram Adjustment -**


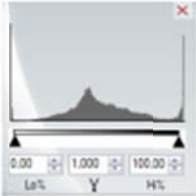



	<b>maximum</b>
--	----------------

**INTERFACE ICONS**

Icon	Features
	<b>Exit</b> To exit the application
	<b>Maximize</b> Maximize application
	<b>Minimize</b> Minimize application
	<b>Create a new patient folder</b> Allows to create a new folder (new patient)
	<b>Open a patient</b> Allows the access to the database of saved patients.
	<b>Open the patient record</b> Allows the access to the folder record showed on the interface and open previously saved files.
	<b>Browse up.</b> Allows sliding images showed in the vertical library.
	<b>Browse down</b> Allows to slide images showed in the vertical library.
	<b>Sequential capture.</b> Defines the number of sequential captures to be done.
	<b>Zoom x1.</b> To show the whole cliché in the viewing window.
	<b>Zoom x2</b> To expand the image
	<b>Compare images</b> Compare images in full screen
	<b>Full Screen</b> Change the viewing mode to full screen
	<b>Save</b> Save image in patient's record

	<b>Dele image</b> Permanently delete the display image				
	<b>Export</b> Allows saving a copy of the image with another name in several images standards.				
	<b>Print</b> Print the image in DICOM or Conventional. <table border="1" data-bbox="584 497 1257 703"> <tr> <td data-bbox="584 497 737 600"></td> <td data-bbox="737 497 1257 600">Print in standard printer</td> </tr> <tr> <td data-bbox="584 600 737 703"></td> <td data-bbox="737 600 1257 703">Print in secondary printer</td> </tr> </table> <p>The setting of the main printer is made in the Setting menu</p>		Print in standard printer		Print in secondary printer
	Print in standard printer				
	Print in secondary printer				
	<b>Edge highlight</b> Improve definition of the edge of the image. Available in 4 levels of density.  Obs: Available only for TIFF images without marking. It is recommended to apply it right after the image capturing.				
	<b>Brightness setting</b> Allows setting the brightness of the image.				
	<b>Inversion of colors</b> Invert the color of the image.				
	<b>Contrast Setting</b> Allows adjusting the contrast of the image.				
	<b>Histogram</b> Allows the setting of the image histogram				
	<b>Gamma Setting</b> Allows the setting of the image gamma				
	<b>Insert text</b> Insert a text on the image.  The definition of color and width is made in the Settings menu				
	<b>Insert Arrow</b> Insert an arrow in the image.  The definition of color and width is made in the Settings menu				


	<p><b>Insert Circle</b> Insert a circle in the image.</p> <p>The definition of color and width is made in the Settings menu</p>				
	<p><b>Insert Freehand drawing</b> Insert a freehand drawing in the image</p> <p>The definition of color and width is made in the Settings menu</p>				
	<p><b>Selection</b> Select an area of the image</p>				
	<p><b>Cut</b> Cut a selected area of the image</p>				
	<p><b>Simple measurement</b> Perform a distance measurement of two points on the image</p>				
	<p><b>Multiple measurement</b> Perform a distance measurement of multiple points on the image</p>				
	<p><b>Angular measurement</b> Perform an angular measurement on the image</p>				
	<p><b>Calibration</b> Perform the calibration of the distance of the image</p>				
	<p><b>Image rotation.</b> Allows 90 degrees rotation of the image.</p>				
	<p><b>Undo</b></p> <table border="1" data-bbox="582 1570 1257 1774"> <tr> <td data-bbox="582 1570 737 1671">  </td> <td data-bbox="737 1570 1257 1671">Undo all</td> </tr> <tr> <td data-bbox="582 1671 737 1774">  </td> <td data-bbox="737 1671 1257 1774">Partial undo</td> </tr> </table>		Undo all		Partial undo
	Undo all				
	Partial undo				
	<p><b>Show/hide toolbar</b> Show/hide toolbar with marking resources/association of the image</p>				
	<p><b>Odontogram</b> Access to Odontogram mode</p>				

	
	<p><b>Histogram window</b> Automatically opens when loading or capturing an image (configurable in the options menu)</p>
	<p><b>Tooth checkbox</b> Used to check the tooth (teeth) showed in the image. Accompany "delete" button to undo checking.</p>
	<p><b>Setting.</b> Allows the access to the parameters of your computer (format and file path of database of images, downloads)</p>
	<p><b>Help</b> Shows tutorial on how to perform an acquisition</p>

**Setting screen**



To set the application, press the icon . The following screen will appear.

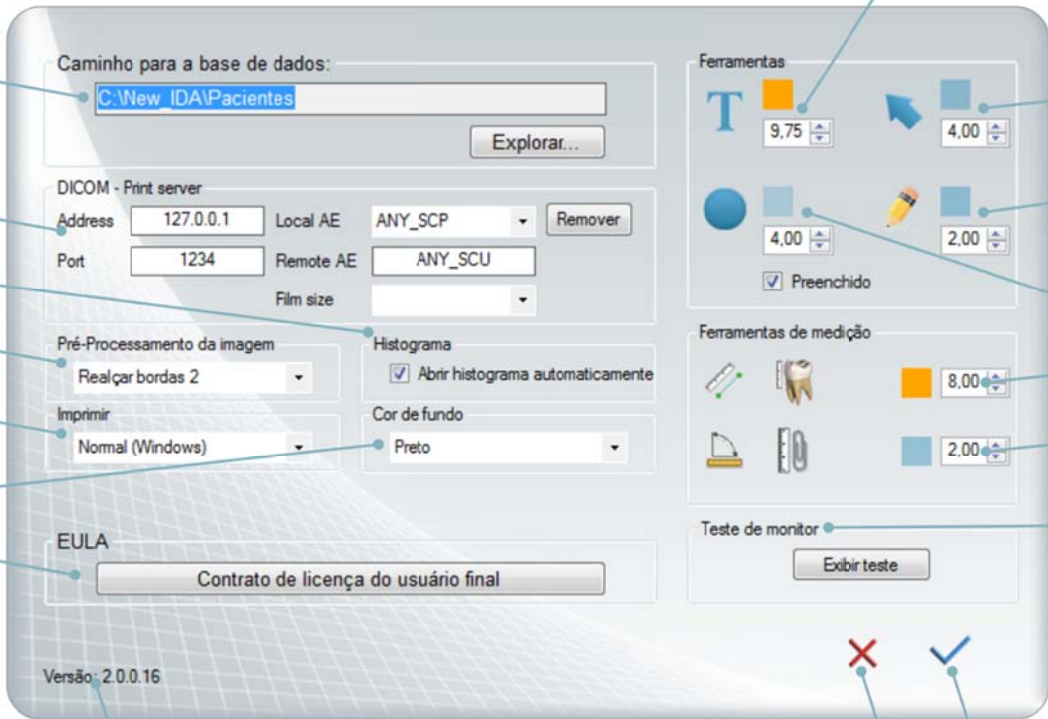

The settings screen contains the following elements:

- 01:** Caminho para a base de dados: C:\New IDA\Pacientes
- 02:** DICOM - Print server Address: 127.0.0.1
- 03:** Imprimir: Normal (Windows)
- 04:** Versão: 2.0.0.16
- 05:** EULA: Contrato de licença do usuário final
- 06:** Ferramentas: T (9.75)
- 07:** Ferramentas: Arrow (4.00)
- 08:** Ferramentas: Circle (4.00)
- 09:** Ferramentas: Eraser (2.00)
- 10:** Ferramentas de medição: Tooth (8.00)
- 11:** Ferramentas de medição: Ruler (2.00)
- 12:** Close button (X)
- 13:** Check button (✓)
- 14:** Pré-Processamento da imagem: Realçar bordas 2
- 15:** Port: 1234
- 16:** Histograma: Abrir histograma automaticamente
- 17:** Teste de monitor: Exibir teste


<b>01 - Path of Database</b> <b>02 - DICOM address</b> <b>03 - Definition of the main printer</b> <b>04 - Software version</b> <b>05 - EULA</b> <b>06 - Color and size of the text*</b> <b>07 - Color and width of the arrow*</b> <b>08 - Color and width of free-hand drawing*</b> <b>09 - Color, width, and filling of the circle.*</b>	<b>10 - Distance measurement: Color and size of the text font.*</b> <b>11 - Angular measurement: Color and size of font.*</b> <b>12 - Cancel changes</b> <b>13 - Confirm changes</b> <b>14 - Preprocessing of image</b> <b>15 - Histogram</b> <b>16 - Background color</b> <b>17 - Monitor test.</b>
---	---


*\*Measurement unit in pixels.*

## Odontogram

The Odontogram resource allows organizing images according to the tooth, building image record to each tooth and generating reports of the treatments.

It's possible to view the image records or start the capture of a new image by clicking the right button of the mouse on the desired tooth.


To access the odontogram, click on the icon . The following screen will appear

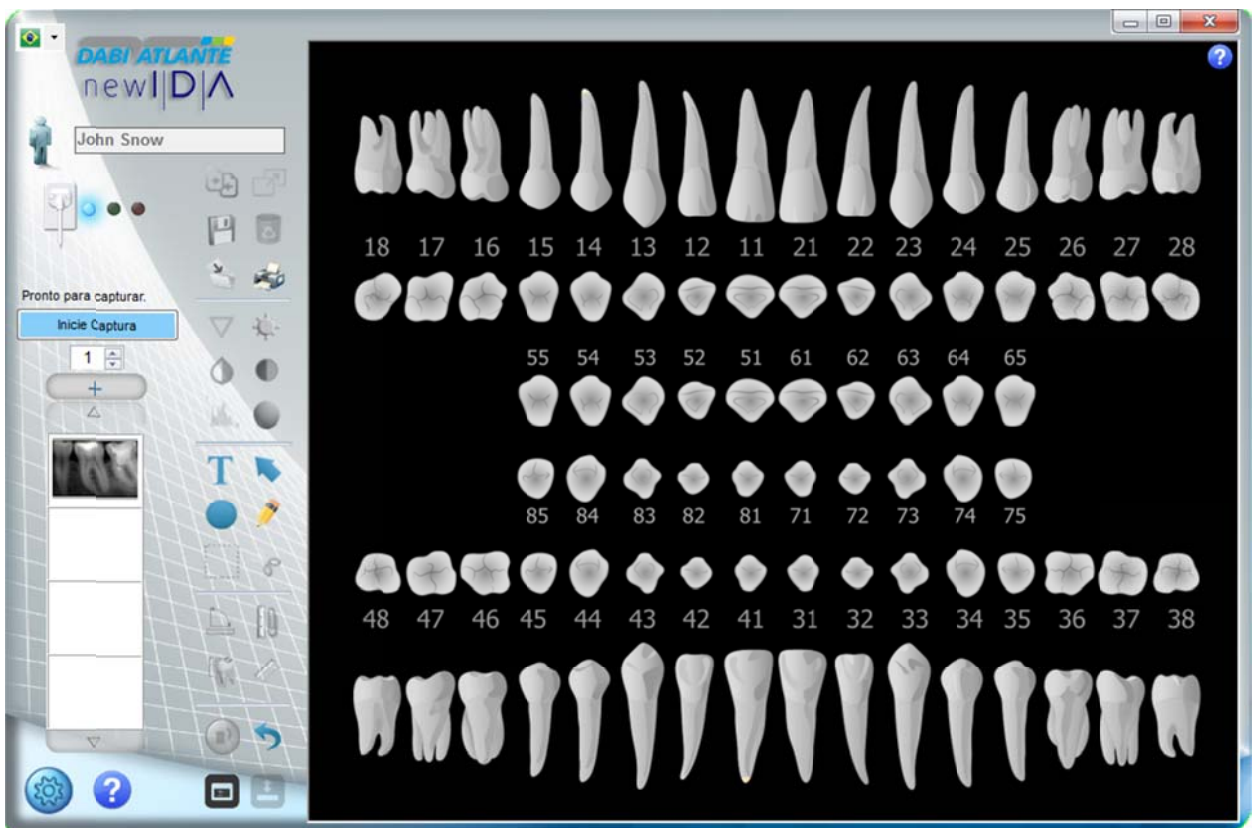


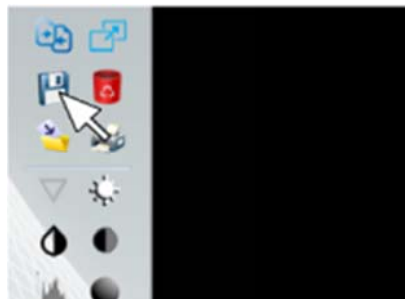
**a) Associate image to the odontogram**

Perform an image capturing


Select numbering of the tooth you want to associate with the image. If you want, insert diagnosis information in the text field.

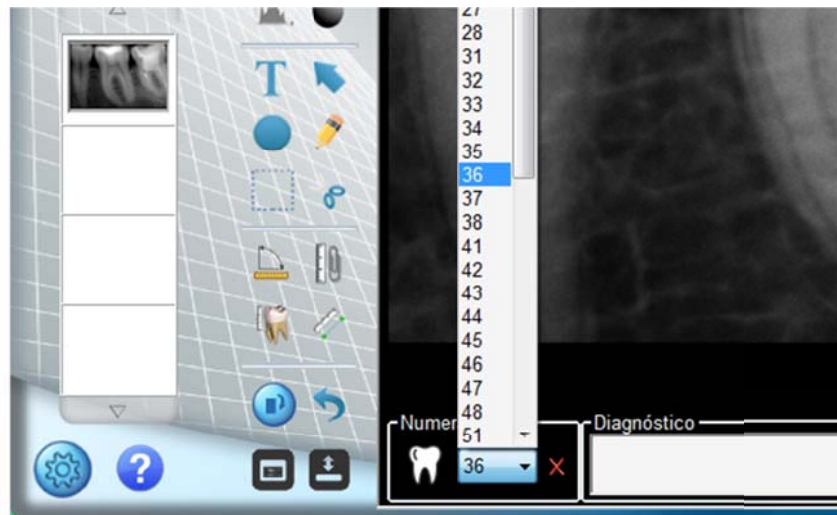

Click on  to associate the image with the patient record.



The tooth whose image is associated will be pointed in YELLOW

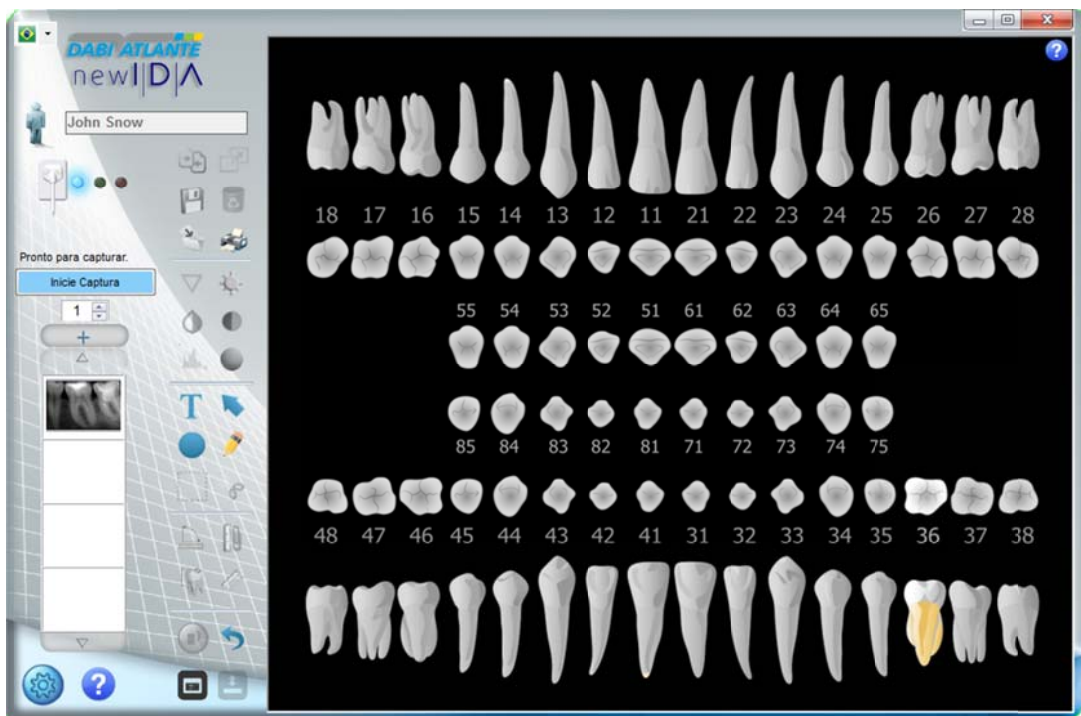


**b) Capture through odontogram**

To start a capture, click the right button of the mouse and after that, on start capture.




### c) *Opening image through odontogram*


Click on a tooth that has an image on the odontogram with the right button of the mouse. Select the day and time of the image you want to open.

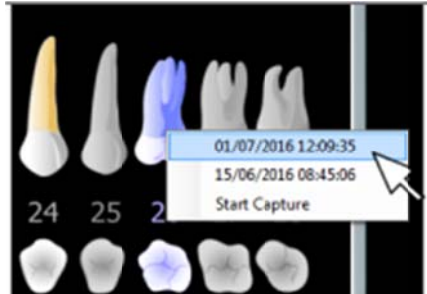

Click with right button to open an image.


### d) *Help odontogram*


The icon  opens a window with an illustrated guide to help with the use of tools on the Odontograma.

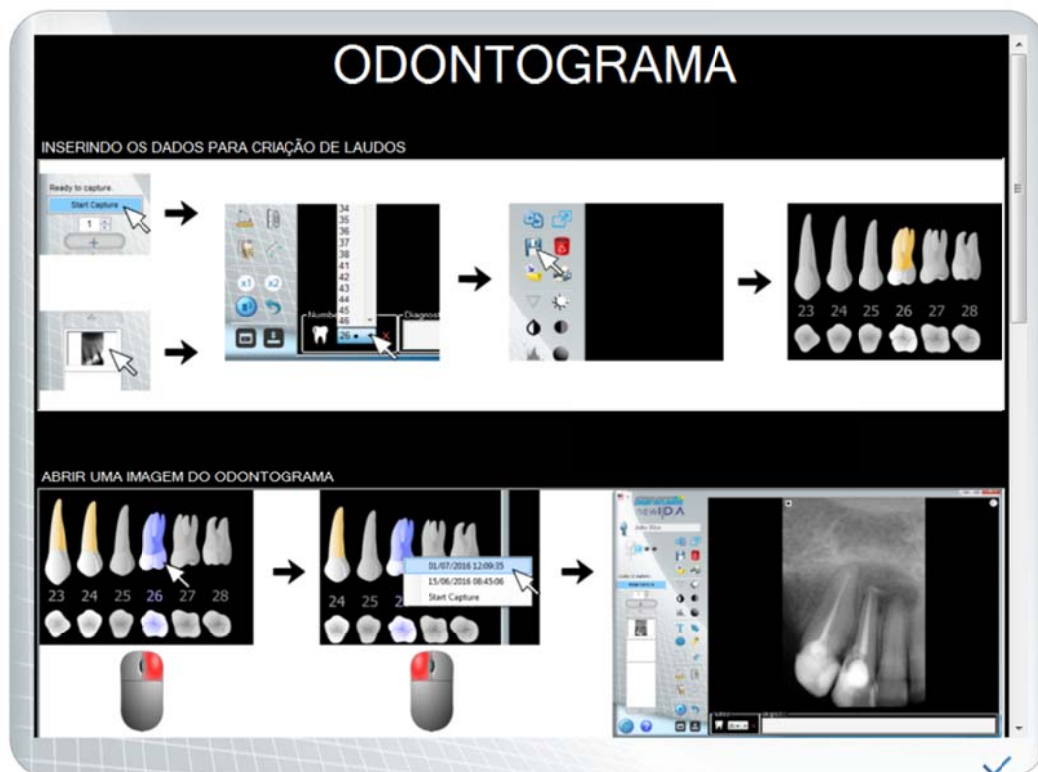

### e) **Generating report through odontogram**



With the odontogram open, click on the icon . The following screen will be displayed.

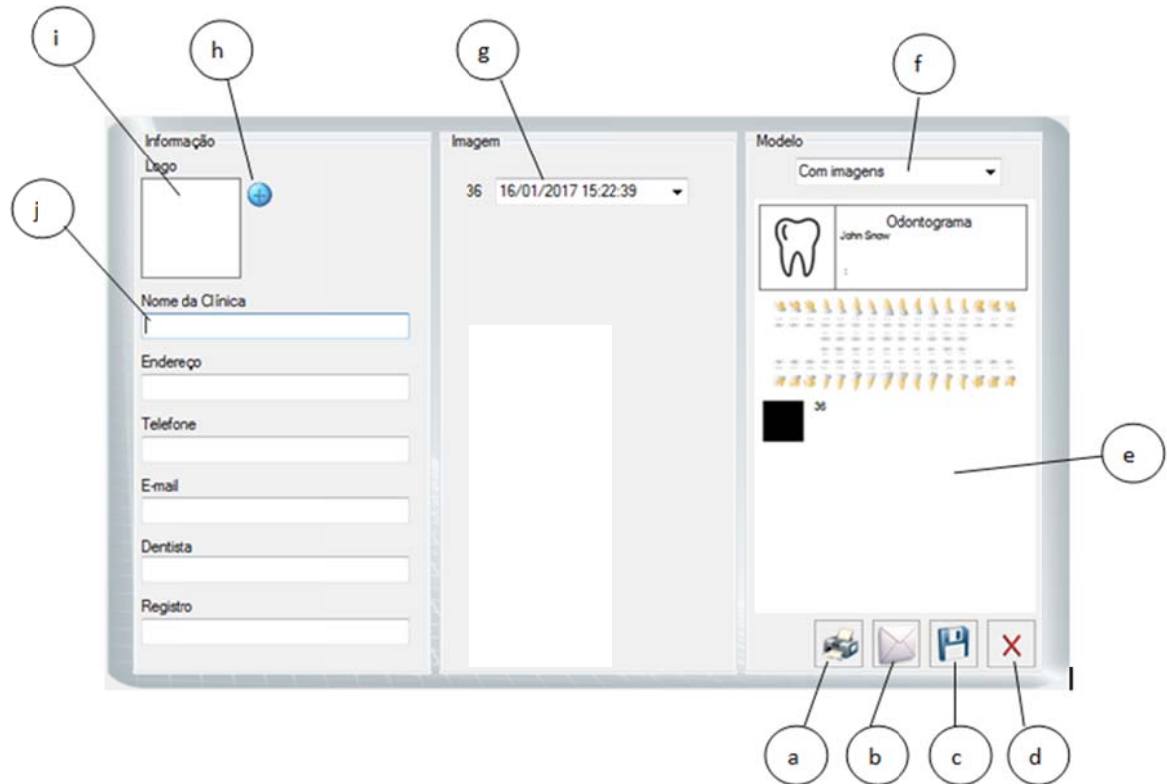

**a - Print report**

**b - Send by email.**

**c - Save report on pdf**

**d - Delete report**

**e - Preview report**

**f - Selection of report model (with images or without images)**

**g - Select the image to be inserted in the report\***

**h - Add logo**

**i - Preview logo**

**j - Clinic data**

\* In case of more than an image associated with the tooth



## CONSUMABLES

Protective plastic covers are used as an additional measure to avoid contamination between patients, users and others.

This equipment should be used with plastic protective covers, specific for use with intraoral dental sensors.


Protective plastic covers are single use consumables that should be discarded for each new patient.

	<p><b>The protective plastic covers don't accompany the product.</b></p>
---	--

	<p><b>When buying a protective plastic cover, make sure they are CE approved, always follow the items listed below:</b></p> <ul style="list-style-type: none"> <li>• <b>Always use protective plastic covers manufactured with non-cytotoxic, non-irritant and non-sensitizing polyethylene film, in accordance with ISO 10993-1.</b></li> <li>• <b>Always use products for exclusive dental use.</b></li> </ul> <p style="text-align: center;"><b>Rx only</b></p> <ul style="list-style-type: none"> <li>• <b>Always use single use products.</b></li> </ul> <p style="text-align: center;"></p>
---	--

**LABELS**

**Identification**

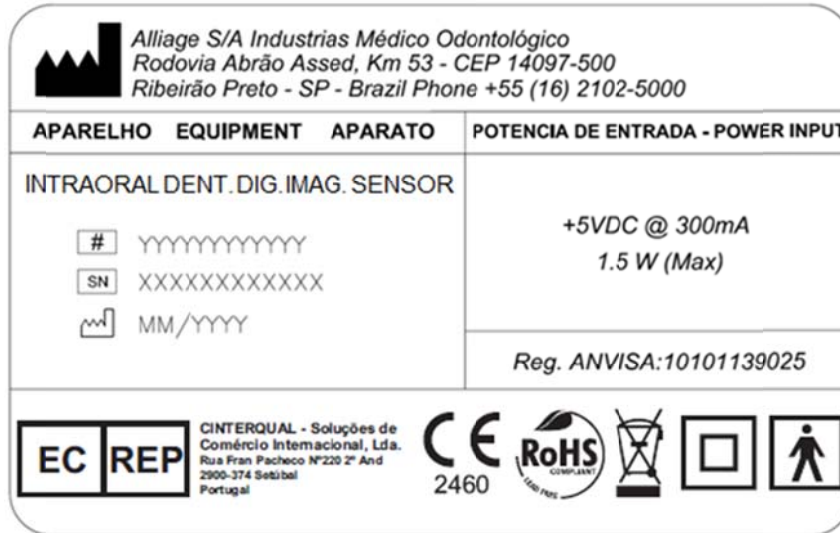

\*DEMONSTRATIVE PICTURE REAL DIMENSIONS 100 x 63 mm

**Safety**



\* DEMONSTRATIVE PICTURE REAL DIMENSIONS 50 x 23 mm

**Refer to instruction manual**



\*DEMONSTRATIVE PICTURE REAL DIMENSIONS DIAM. 10 mm

**INTRAORAL RADIOGRAPHY EQUIPMENT**

The **Intraoral Dental Digital Imaging Sensor** is compatible with intraoral x-ray equipment with electronic timer that complies with current standards EN 60601-1, EN 60601-1-2, EN 60601-1-3 and IEC 60601-2 -65 and meets the following requirements:


- The intraoral X-ray generator must operate at a voltage between 60 and 70 kV and an anode current of 7mA.
- The intraoral X-ray generator must be installed according to the manufacturer's instructions and requirements.



**Don't use generators with power of less than 60KV.**

## SPECIFICATIONS AND TECHNICAL CHARACTERISTICS

### General information

	Alliage S/A Industrias Médico Odontológica	
	<b>Telephone:</b> +55 (16) 3512-1212	
<b>Address:</b> Rodovia Abrão Assed, Km 53 - Recreio Anhangüera Ribeirão Preto – SP - Brasil CEP 14097-500		
<b>Name</b>	<b>Intraoral Dental Digital Imaging Sensor</b>	
<b>Model</b>	<b>New IDA</b>	
Equipment Classification		
Classification class according to ANVISA	Class I	
Classification class according to CE	Class IIa	
Classification of the Equipment according to the NBR IEC 60601-1 regulation		
Classification of the product for applied parts	Applied parts BF type	
Protection Against Electrical Shock	Class II	
Protection Against Harmful Water Infiltration	IP00 - This product is not protected against harmful water infiltrations and particulate matter.	
Degree of application security in presence of one flammable anesthetic mixture with the air, oxygen or nitrous oxide	Inappropriate equipment	
Operation Mode	Continuous operation	
Input Voltage	+5 Vdc	
Input Current	300 mA	
Power Consumption	1.5 W	
Cable length	3 m	

### Characteristics of the sensor

Size*	0	1	2
Sensor resolution (pl/mm)	25	25	25
External dimensions (mm)	21.7 x 30.3	24.7 x 38.3	30.6 x 44.2
Active surface dimensions (mm)	17 x 22	20 x 30	26 x 36
Matrix dimension	850 x 1100	1000 x 1500	1300 x 1800
Pixels number (MegaPixels)	0.93	1.50	2.34
Technology	CMOS	CMOS	CMOS
Gray level	4096	4096	4096
Image resolution	12 bits	12 bits	12 bits
Image format	TIFF, JPG, PNG, BMP DICOM	TIFF, JPG, PNG, BMP DICOM	TIFF, JPG, PNG, BMP DICOM
Optical Fiber	1	1	1
Weight	60 g	65 g	70 g
Connection	USB2 - High speed	USB2 - High speed	USB2 - High speed
Cable length	3 meters	3 meters	3 meters

\* Order on request

***Environmental conditions for transportation and storage***

Room temperature range for transportation and storage	+4°C to +40°C
Relative humidity range for transportation and storage	< 85% RH
Atmospheric pressure band	797 hPa to 1013 hPa (598 mmHg to 760 mmHg)

***Conditions for installation and operation of the equipment***

Operating room temperature band	+10°C to +30°C
Operating relative humidity range (non-condensing)	< 75% RH
Atmospheric pressure band	797 hPa to 1013 hPa (598 mmHg to 760 mmHg)
Operating altitude	≤ 2000 m

## **CERTIFICATIONS AND STANDARDS**

This product bears CE mark in accordance with the provisions of Council Directive 93/42/ EC of June 14, 1993 on medical devices as amended by Directive 2007/47/EC of September 5, 2007.

The **Intraoral Dental Digital Imaging Sensor**, along with the components of his system, have been evaluated and are conformed to the international EMC, safety and quality standards listed below, among others:

### ***EMC / Security***

- EN 60601-1: 2006 / A1: 2013 - Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.
- EN 60601-1-2: 2015 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests.

### ***Usability***

- EN 60601-1-6: 2010 - Medical electrical equipment - Part 1-6: General requirements for basic safety and essential performance - Collateral standard: Usability
- EN 62366: 2008 - Medical devices - Application of usability engineering to medical devices.

### ***Software***

- EN 62304: 2006 / AC: 2008 - Medical device software - Software life-cycle processes.



### ***Quality***

- EN ISO 13485: 2016 - Medical devices - Quality management systems - Requirements for regulatory purposes
- EN ISO 14971: 2012 - Medical devices - Application of risk management to medical devices.

## SYSTEM OPERATION



### PREPARATION FOR USE

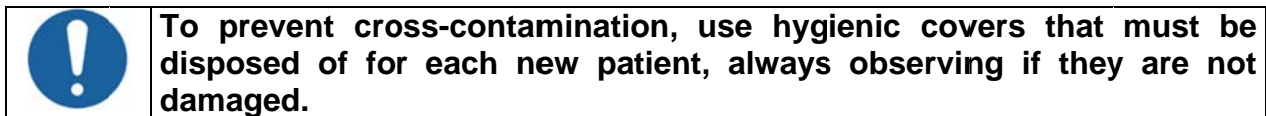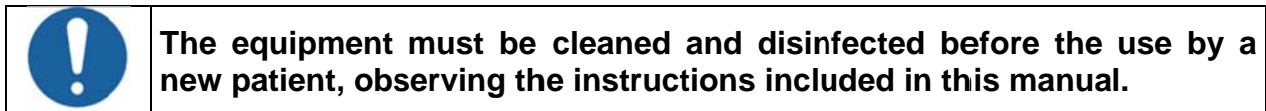



Cover the sensor with the disposable plastic cover, also covering the cable to ensure that it will not be in contact with the patient.



The sensor must be positioned manually.

In the case of manual placing, introduce the sensor in the mouth of the patient in the deeper possible part.

The active surface of the sensor must be placed towards the source of X-Rays.

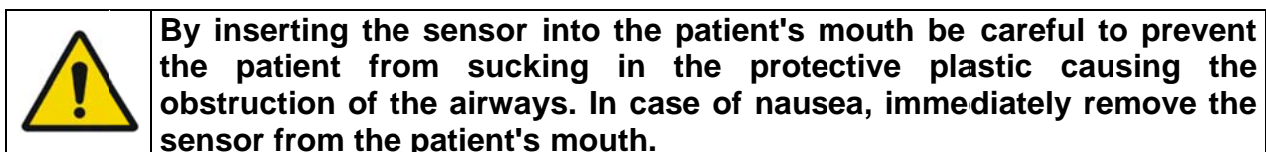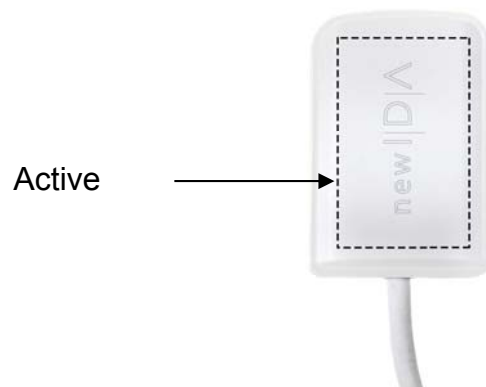




**During the examination, take the necessary precautions to prevent the patient from biting the sensor cable.**

### **ACQUISITION OF RADIOGRAPHIC IMAGES**


To use the Intraoral dental digital imaging sensor New IDA, start the computer and verify if it's on.


Start the software of image capturing clicking on the shortcut icon on the Desktop or Start → All programs → New IDA

The home screen of the application will appear

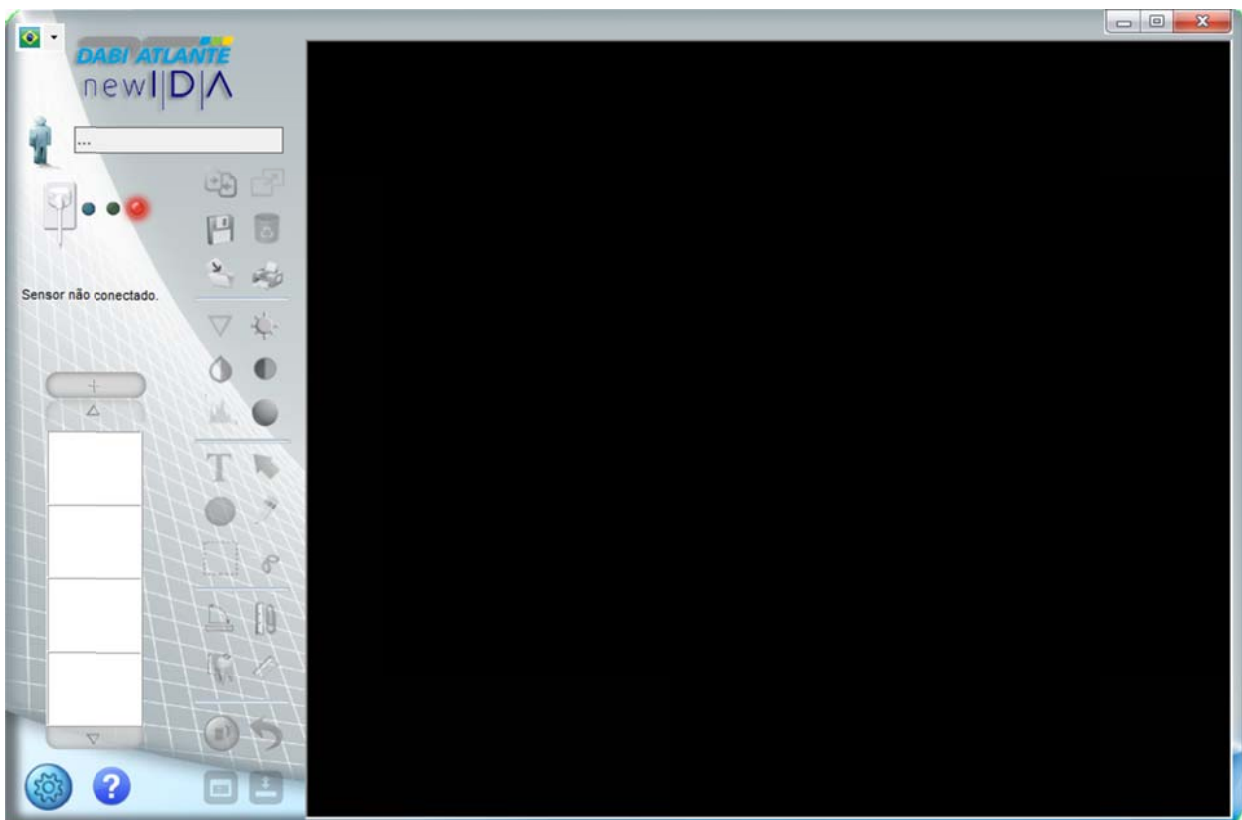

### Create a new registration of patient

Click on icon  the next screen will be shown

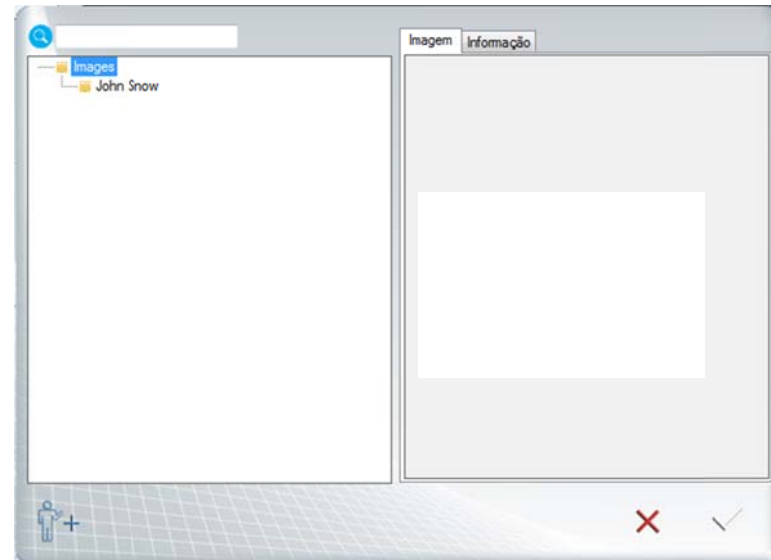

Next, click on icon 

Fill the fields to do the registration of the new patient (Name and Last name are obligatory fields). Next, click "OK"

ID do paciente	
<input type="text"/>	
Digite o nome do paciente:	
<input type="text"/>	
Sobrenome	
<input type="text"/>	
Sexo	Data nasc
<input type="text"/>	<input type="text"/>
Documento	
Tipo	Numeração
<input type="text"/>	<input type="text"/>
Telefone	E-mail
<input type="text"/>	<input type="text"/>
Endereço	
<input type="text"/>	
<input type="button" value="OK"/> <input type="button" value="Cancelar"/>	

Select the file that you created and click OK.

The name of the folder appears in the capture window. The images will be stored in this folder.



### **Opening a patient registration**




Click on the icon  
Select the patient directory that you want to open.

Click on  to confirm or click on  to go back to the initial interface.

### **Changing a patient registration**






Click on the icon  
Select the patient directory that you want to open.  
Click on the information tab and fill the fields you want to change.

Click on  to save the changes.

Click on  to confirm or click on  to go back to the initial interface.

### **Sensor status**


Symbol	Color	Sensor status
--------	-------	---------------

	Blue icon	The sensor is connected and was recognized by the application. Waiting for Start command to activate the sensor.
	Green icon	The sensor is active and able to receive X-Rays.
	Red icon	The sensor is not connected, canceled acquisition, patient folder not selected or sensor error.

### ***Process of image capturing***

Verify if the sensor status has the **BLUE** light.

Select the necessary exposition time on the timer of the periapical X-Ray.

	<b>Selecting the X-Ray exposition setting, verify if they are all appropriate</b>
	<b>The setting depends on multiple variables, such as... the X-Ray tube, patient anatomy, distance between the sensor-front and location of the sensor in the oral cavity.</b>
	<b>As a reference, it must be considered that the New IDA sensor requires an 85% dose less that the movie "Tipo-D".</b>

On the application interface, click on **INITIATE CAPTURE**





The icon of the sensor turns **GREEN**. The sensor will be activated and waiting for the X-Ray exposure, being kept active during 120 seconds. An indicative progress bar shows the counter.



At the end of this period, if there`s no X-Ray emission, the sensor will be back to standby mode. Just click START CAPTURE to reactivate it.


The activation of the sensor can be interrupt at any moment, just by clicking CANCEL CAPTURE.

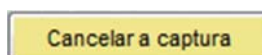



	<b>Before the shot, to avoid unnecessary X-Ray exposure to the patient, make sure that the sensor status shows a GREEN icon and has the message "Waiting to expose" in it.</b>
---	--

Perform the exposure with the X-Rays generator to capture the image.

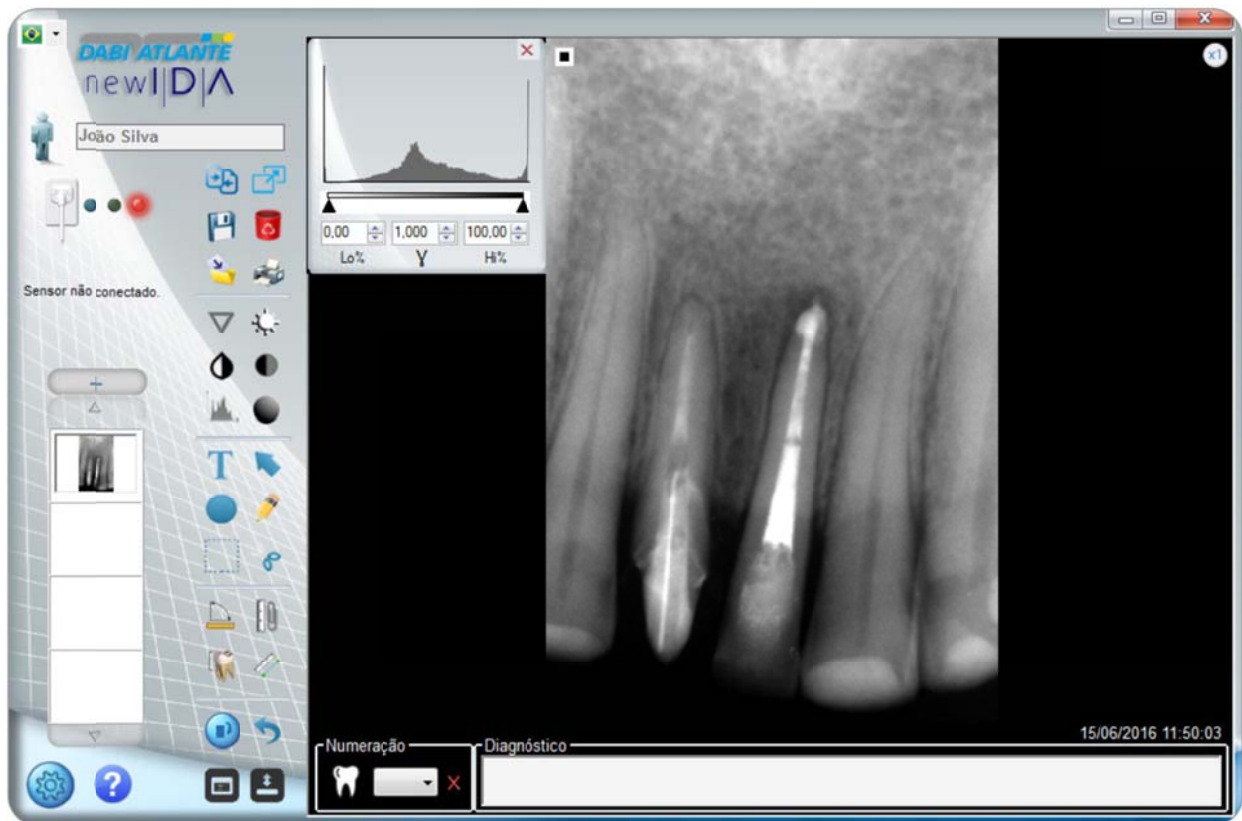After the exposure, the image will appear on the screen of the application.




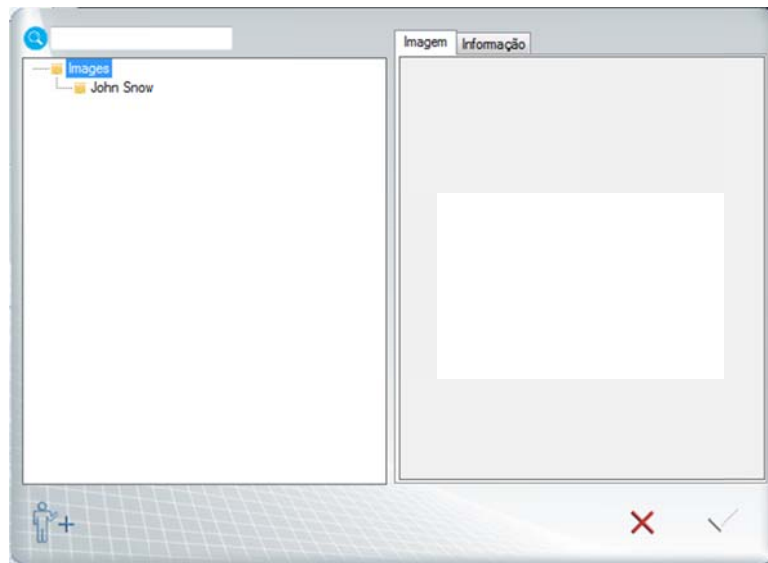
If the captured image is suitable for diagnosis, click the icon  to save it in the patient folder, otherwise, delete the image clicking on the icon  and perform new exposure.


At the end of the exam, remove the sensor from the mouth of the patient and dispose the plastic protection following the recommendations included in this manual.

	<b>Keep the sensor in its box when it's not in use.</b>
---	---

### ***Images of the patient's registration***

To access the registered images of a patient, open the register of him clicking on the  icon the following screen will appear




Select the patient folder and after, click on the icon . The following screen will be displayed.




All images with date information and time of the capture will be exhibited.

**a) Add a new image to the register**


To add a new image to the patient register, click on the icon . A search window will be opened.

Select the image you want to add to the register and click "Open".

**b) Delete an image from the register**

To delete an image, select the image you want and click  to delete it permanently.

**c) Open an image from the register**

To open an image from the register, select the image that you want to open and click on the icon. 

**Metric Calibration**

	<b>The process of metric calibration must be performed at least one time after installation.</b>
---	--


	<b>The metric calibration depends on the radiographic technique used. The metric technique must be performed every time the technique is changed.</b>
---	---

The intraoral calibration is unique for any device and it depends on the radiographic technique used. There are two basic periapical radiographic techniques: the Bisector Technique and Parallelism Technique

**a) Radiographic techniques**

Parallelism Technique or Long cone


In this technique, the digital sensor is placed in parallel to the axis plan of the tooth and the focal point of the x-ray generator must be positioned at a distance around 40cm of the sensor.

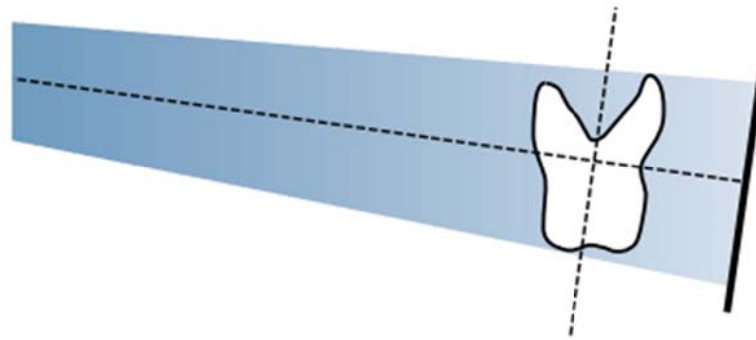

This technique produces radiograph with minimum distortion, showing objects that are being x-rayed in their true anatomic relation and size.

### *Bisector Technique or Short Cone*


In this technique, the digital sensor is placed in a way that the beam of the central X-Ray must fall upon in a perpendicular way over an imaginary plan of the bisector of the angle formed between the long axis of the tooth and the sensor capture surface, according to the isometric law from Cieszinski. For this, the focal point of the X-Ray generator must be placed at a distance of approximately 20cm of the sensor.

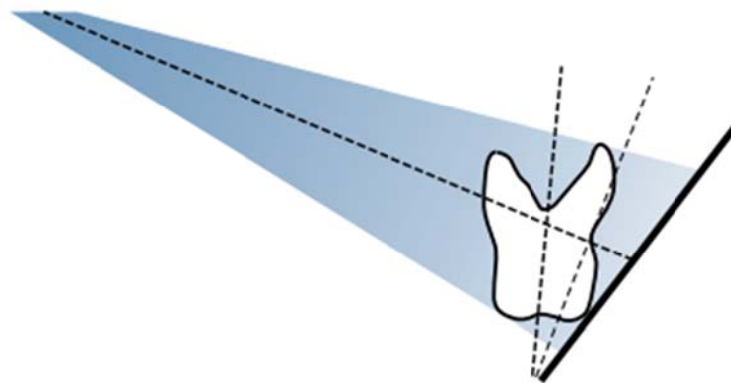


This technique produces radiographs with distortion, regardless of the technique and precision of the operated. However, it also provides a relatively simple, fast and comfortable positioning to the patient.

### ***b) Metric calibration procedure***


To perform the metric calibration, first perform the capture of an image of an object that can be physically measured, using the radiographic technique that will be used with the sensor.

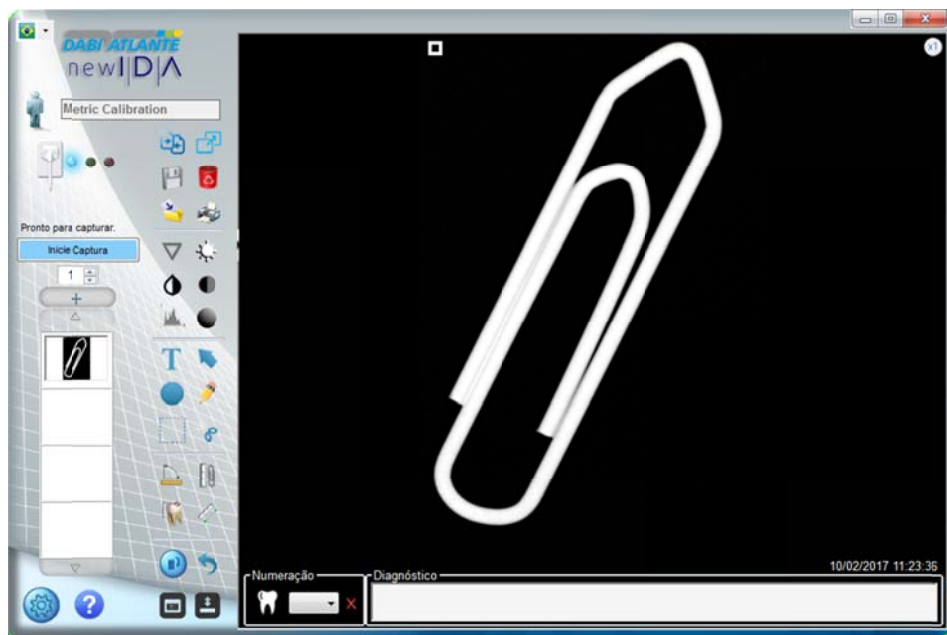

After this phase, click on Calibration icon.  
Draw a line between the two points that will be measured later.

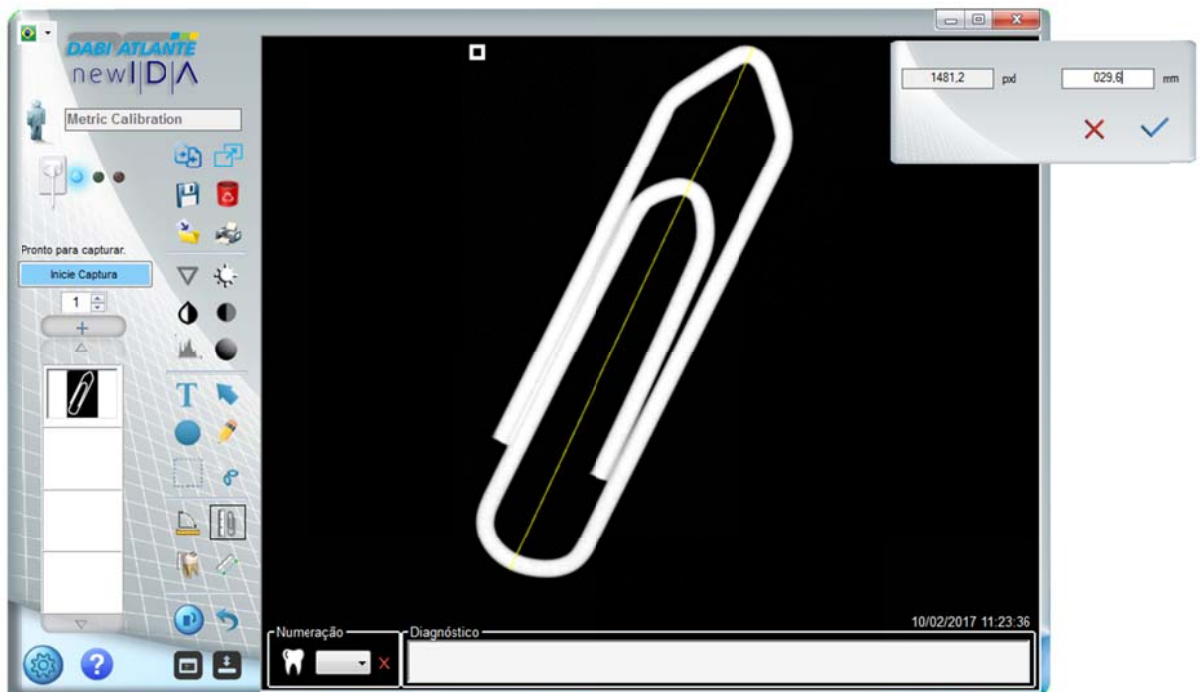


The system will show a standard distance in pixel and mm.

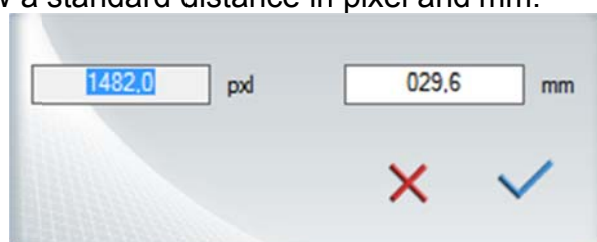

Perform the physical measurement of the x-rayed object.



Type the distance measured in mm.

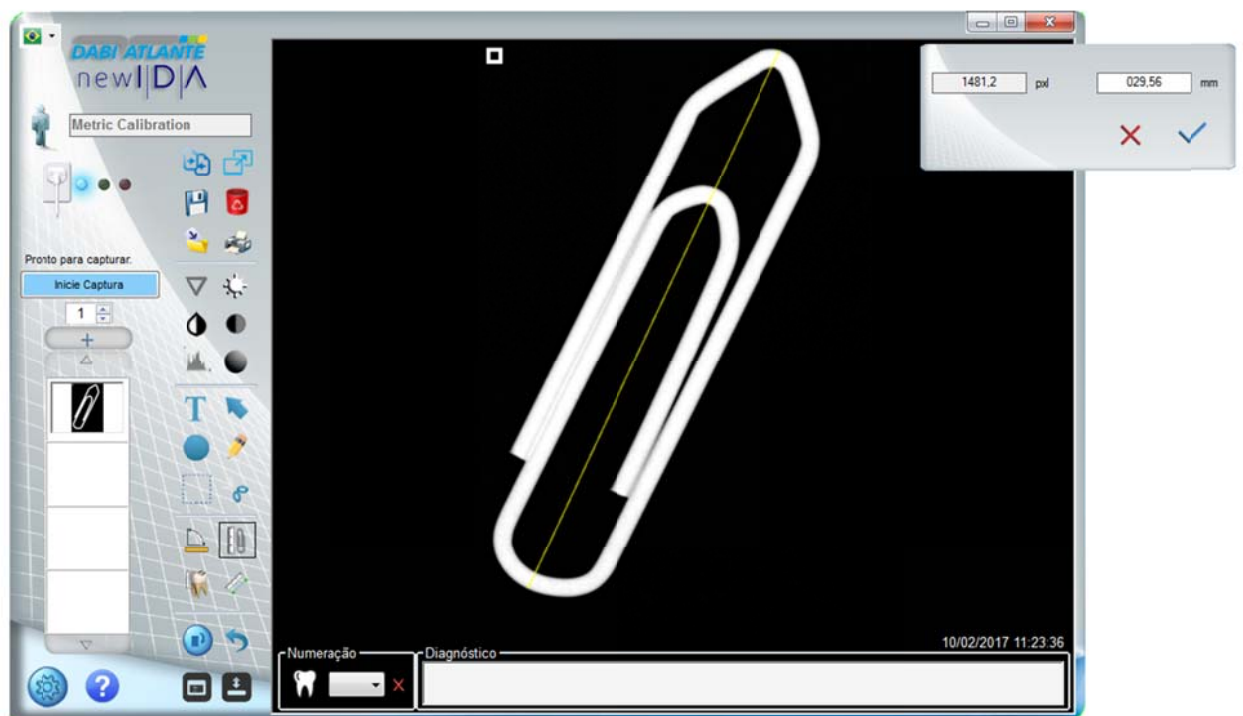

After entering the distance, click on .

### ***Precision of the measurement***

Parameter	Range	Precision
Distance	0 to 99,9 mm	0,1mm
Angle	0,00 to 179,99°	0,01°

## RECOMMENDATION FOR EXAMS

The radiographs must be performed only when there's a diagnosis expectation that will affect the treatment of the patient. The dentist shall consider the benefits of getting radiographs against the risk of exposing the patient to radiation.

Due to the effects of radiation accumulation over time, all efforts must be done to minimize the patient exposure.


Use lead apron and thyroid collar,

Use pediatric profile or low usage and select the lowest permissible time of exposure.

There may be clinical circumstances for which a radiography is indicated, but a diagnosis image can't be obtained. For example, the patient may not be able to cooperate with the dentist.

PATIENT AGE AND STAGE OF DENTAL DEVELOPMENT <sup>1</sup>					
TYPE OF CONSULTATION	Children with primary dentition (before the eruption of the first permanent tooth)	Children with transitory dentition (after the eruption of the first permanent tooth)	Teenager with permanent dentition (before the eruption of the third molars)	Adult toothed or partially edentulous	Adult, Edentulous
New Patient * being evaluated for oral diseases	Individualized radiographic exam which consists of periapical / Occlusal view and/or posterior bitewings if the proximal surface can't be viewed or probed. Patients with no indication of the disease and with open proximal contacts may not require a radiography	Individualized radiographic exam which consists of posterior bitewings with panoramic exam or posterior bitewings and selected periapical images.	Individualized radiographic exam which consists of posterior bitewings with panoramic exam or posterior bitewings and selected periapical images. A complete intraoral radiography exam from the mouth is preferable when the patient has clinical evidence of generalized oral disease or an extensive record of dental treatment.		Individualizes radiography exam, based on signs and clinical symptoms.
Patient follow-up* with decay or increased risk of decay**	Posterior bitewing exam in intervals of 6-12 months if the proximal surfaces can't be visually examined or with a probe			Posterior bitewing exam in intervals of 6-18 months	Non-applicable
Patient follow-up* without decay or increased risk of decay**	Posterior bitewing exam in intervals of 12-24 months if the proximal surfaces can't be visually examined or with a probe		Posterior bitewing exam in intervals of 18-36 months	Posterior bitewing exam in intervals of 24-36 months	Non-applicable
Patient follow-up* with periodontal disease	Clinical judgment related to the need and type of radiographic images for the evaluation of the periodontal disease. The image can consist, but not be limited to, bitewing and/or periapical images from areas where the periodontal disease (except non-specific gingivitis) can be clinically demonstrated.				Non-applicable
Patient (New and Return)	A clinical judgment about the need	A	clinical	It is usually	not indicated for

<p><b>for growth monitoring and dentofacial development, and/or evaluation of the dental/skeletal relations</b></p>	<p>and type of radiographic images for evaluation and/or monitoring of the growth and development or evaluation of dental and skeletal dentofacial relations.</p>	<p>judgment about the need and type of radiographic images for evaluation and/or monitoring of the growth and dentofacial development or evaluation of dental and skeletal relations. Panoramic or periapical exam to evaluate the development of the third molars</p>	<p>growth and development monitoring. A clinical judgment about the need and type of radiographic images for evaluation of dental and skeletal relations.</p>
<p><b>Patient with other circumstances including, but not limited to, proposed or existent implant, other dental and craniofacial pathologies, restoration / endodontic care, treatment for periodontal diseases and remineralization of the decay</b></p>	<p>Clinical judgment about the need and type of radiographic images for evaluation and/or control of these conditions</p>		

	<p><b>This recommendation is subjected to the clinical judgment and is not applicable to every patient.</b></p> <p><b>It is the dentist responsibility to follow the ALARA principle (as low as reasonably possible) to minimize the exposure of the patient.</b></p>
---	---

**\* Clinical situations in which the radiograph is indicated includes, but it's not limited to:**

**A. History of positive discoveries**

1. Previous periodontal or endodontic treatment
2. History of pain or trauma
3. Family history of dental abnormalities
4. Post-surgery evaluation for healing
5. Remineralization monitoring
6. Implant presence, pathology related to previous implants or evaluation for implant placement

**B. Clinical positive - Signs and Symptoms**

1. Clinical evidence of periodontal
14. Pain or ATM dysfunction

- disease
2. Big or deep restoration
  3. Injury from deep cavity
  4. Crooked or broken teeth
  5. Swelling
  6. Evidence of dental or facial trauma
  7. Teeth mobility
  8. Fistula
  9. Clinical suspect of sinus pathology
  10. Growth anomalies
  11. Oral involvement in the known or suspicious systemic disease
  12. Neurological positive findings in the head or neck
  13. Evidence of strange objects
  15. Facial asymmetry
  16. Pillar for prosthesis partially removable or fix
  17. Unexplained bleeding
  18. Inexplicable teeth sensibility
  19. Unusual eruption, spacing or teeth migration
  20. Morphology of the unusual tooth, calcification or color
  21. Inexplicable absence of teeth
  22. Clinical Acid Erosion
  23. Peri-implanting


**\*\*The factors that increase the risk for caries can be available using the ADA forms for caries risk assessment (0 - 6 years old and older than 6 years old).**


<sup>1</sup>U.S. Department of Health and Human Services. **Dental Radiographic Examinations: Recommendations for Patient Selection and Limiting Radiation Exposure.** Available in [http://www.ada.org/~media/ADA/Member%20Center/Files/Dental\\_Radiographic\\_Examinations\\_2012.aspx](http://www.ada.org/~media/ADA/Member%20Center/Files/Dental_Radiographic_Examinations_2012.aspx). Accessed on November 2, 2015.

<sup>2</sup>The American Academy of Pediatric Dentistry. **Guideline on Prescribing Dental Radiographs for Infants, Children, Adolescents, and Persons with Special Health Care Needs.** Available in [http://www.aapd.org/media/policies\\_guidelines/e\\_radiographs.pdf](http://www.aapd.org/media/policies_guidelines/e_radiographs.pdf). Accessed on November 2, 2015.

<sup>3</sup>U.S. Department of Health and Human Services. **Pediatric X-ray Imaging** Available at: <http://www.fda.gov/Radiation-EmittingProducts/RadiationEmittingProductsandProcedures/MedicalImaging/ucm298899.htm>. Accessed on November 2, 2015.

## CLEANING AND DISINFECTION

	<b>Before starting the cleaning and disinfection procedure, disconnect the sensor from the USB port of the computer to avoid damage to your equipment.</b>
---	--

	<b>For your protection, during the cleaning and disinfection process, it's mandatory the use of IPEs such as disposable gloves and safety glasses.</b>
---	--

The cleaning and disinfection process must be performed at each patient change and before the storage of the product.

When starting the process, check the existence of visible dirt, such as blood or saliva.

Clean carefully the New IDA sensor and the first 30cm of the USB cable.


For the cleaning, use a clean and soft cloth damp with neutral soap and then perform the drying with a clean and soft cloth or with a paper towel.

For the disinfection process of the New IDA sensor, use disinfectants towels that have active components based on *Didecyldimethylammonium chloride* respecting the contact time indicated by the manufacturer.


There's no cycle limits or time of application that the New IDA sensor and its parts can tolerate during the cleaning and disinfection process, following the instructions of this manual.

	<b>Don't use sanitizing or cleaning agents as a spray.</b>
---	--

	<b>Don't dip the New IDA sensor in liquid sanitizing.</b>
---	---

	<b>The New IDA sensor cannot be sterilized.</b>
---	---

## MAINTENANCE

	<b>The New IDA sensor or any of its parts can't receive maintenance or assistance during its use with a patient.</b>
---	--

### ***PERIODIC INSPECTION***

It is mandatory that this equipment be inspected regularly to ensure the operational safety and functional reliability.

The periodical inspection must be performed at regular intervals (at least once a month) to ensure that the product is permanently safe and operational. All components subject to the normal wear must be verified and, if necessary, replaced.

The manufacturer is exempt from liability for uncommon results, in case the user don't perform the maintenance recommended by him.

Inspection and service are not part of the warranty of the equipment.

The inspections performed must be documented and kept with the equipment.

The following table shows a description of the main inspection items and recommended frequency.

<b>Item</b>	<b>Control description</b>
USB cable	Connection / Physical damage
Intraoral sensor	Communication / Operation / Overheating / Physical Damage
Accessories	General damage that can cause risk

In case are detected problems during the inspections, contact us, with the Alliage Service Department.

### ***PREVENTIVE MAINTENANCE***

Besides the periodical inspection, to ensure a long durability and good functioning of your equipment, it's important to perform a preventive maintenance for a period of at most three (3) years.

Contact the Alliage Service Department for our periodical review and program of preventive maintenance.

### ***CORRECTIVE MAINTENANCE***

All instructions about the use of the equipment as intended are provided in this user manual. If a problem is detected and can't be corrected with the instructions in the error diagnosis section, contact Alliage Service Department.

Alliage states that the supply of circuit diagrams, components lists or any other information that provides technical assistance for the user can be requested if previously agreed between him and Alliage Enterprise.

The warrant will be annulled if the original parts are removed/replaced by technicians from non authorized services.



**The corrective maintenance that can be performed by the Intraoral dental digital imaging sensor New IDA user is limited to the replacement of the USB cable.**

## ERROR DIAGNOSIS

In the case of problems with the functioning of your sensor, verify the following:

Symptoms	Diagnosis and solutions
Red status of the sensor, indicating that it is not connected	<p>The sensor was not recognized by the computer or it is off.</p> <ul style="list-style-type: none"> <li>- Verify if the USB cable is properly connected to the computer</li> <li>- Restart the computer</li> <li>- Verify if the drivers of the sensor are correctly installed.</li> </ul>
Red status of the sensor, indicating that it is not connected And start capture button doesn't appear	<p>Patient folder was not selected</p>
Blue status of the sensor and there's no image.	<ul style="list-style-type: none"> <li>- Make sure the sensor is activated (button "Start capture" before the X-Ray shots.</li> <li>- Verify the correct performance of the generator. <ul style="list-style-type: none"> <li>- Make sure the sensor surface is oriented to the x-rays source and that is correctly aligned.</li> </ul> </li> </ul>
Green status of the sensor and there's no image.	<ul style="list-style-type: none"> <li>- Verify the correct performance of the X-Ray generator.</li> <li>- Make sure the active sensor surface is oriented to the x-rays source and that is correctly aligned.</li> </ul>
Error "TransfersThread() - pre-capture(0) SafeHandle was closed"	<p>Fail in USB communication during the capture of the sensor.</p> <ul style="list-style-type: none"> <li>- Verify if the USB cable shows any damage indicative</li> <li>- Verify USB connection with the computer</li> <li>- Verify if the USB port of the computer has the capacity to supply enough power for the proper operation of the sensor.</li> </ul>
<p>Error "New IDA has stopped working</p> <p>A problem made the program stop working correctly. Windows will close the program and will notify if there's an available solution"</p>	<p>Fail in USB communication during the setting of the sensor.</p> <ul style="list-style-type: none"> <li>- Verify if the USB cable shows any damage indicative</li> <li>- Verify USB connection with the computer</li> <li>- Verify if the USB port of the computer has the capacity to supply enough power for the proper operation of the sensor.</li> </ul>
	<p>Fail in RAM memory</p> <ul style="list-style-type: none"> <li>- Verify the RAM memory of the computer</li> </ul>

Warning "There's a difference between the serial number of the New IDA sensor and the calibration files. If the calibration files belongs to this sensor, please ignore this message"	Calibration file doesn't belong to the sensor - Contact the technical service
	Calibration file belongs to the sensor, but it was generated with a wrong serial number. - Ignore warning
Error "To run this application, it is necessary to install one of the following versions of .NET Framework v4.0"	Fail to install.NET 4.5.1. - Install Microsoft.NET 4.5.1 or reinstall software
Very clear image	- Very long exposure time. - Adjustment of your monitor. (brightness and contrast)
Very dark image	- Insufficient exposure time. - The tension of the generator is insufficient (<60kV). - Adjustment of your monitor. (brightness and contrast)
Image completely dark	- The active area of the sensor was not exposed to the X-Rays - Insufficient X-Ray dose
Image with artifacts or lines	Calibration file doesn't belong to the sensor - Contact the technical support
Image failure	USB cable failure - Verify if the USB cable shows any damage indicative - Verify USB connection with the computer - Verify if the USB port of the computer has the capacity to supply enough power for the proper operation of the sensor.
	Fail in the sensor due to physical damage (impact) - Contact the technical service

In the case of pending issues, contact Alliage authorized technical service.

### **CABLE REPLACEMENT**

	<b>Before starting the cable replacement procedure, disconnect the sensor from the USB port of the computer to avoid damage to your equipment.</b>
---	--



The electronic circuits of Intraoral dental digital imaging sensor are susceptible to electrostatic discharge (ESD). During the cable replacement, measures must be taken to protect the device against possible damage by ESD.

With the help of a screwdriver with the tip of 3 x 0,8mm, remove the finishing cover of the screw.




With the screwdriver with the tip of 3 x 0,8mm, remove the screws that attach the cable.



Remove the USB cable and perform the replacement.



Attach the screws again.

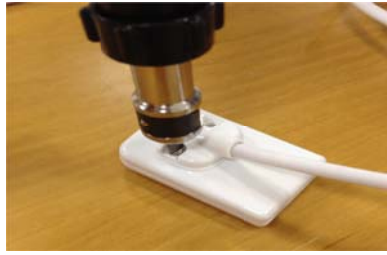

**It is recommended the use of a torquemeter adjusted for 10 N.cm**

Make sure there's no space between the plastic parts.



**Incorreto**



**Correto**

Place the cover of the screws available in the kit and press them until they fit in the plug.



### ***AUTHORIZED SERVICE NETWORK***

All services performed in the Alliage equipment must be done by an Authorized Technical Assistant, because, otherwise, they won't be covered by the warrant.

If you need to request electrical schemes and/or components specifications that are not declared in the user manual, use the Alliage Customer Attendance Service to make the request.

Telephone: +55 (16) 3512-1212

E-mail: [imagem.eagle@alliage-global.com](mailto:imagem.eagle@alliage-global.com)

Address: Rodovia Abraão Assed, Km 53 - Recreio Anhangüera - Ribeirão Preto-SP/  
Brasil CEP 14097-500

### ***WARRANTY TERM***

The Warranty Term, in three copies, must be properly filled by the Alliage Authorized Technical Assistant. One of the copies, already marked, will be delivered to the owner for confirmation of the equipment under warranty. One of the remaining copies will stay with the distributor and the other one will be sent to the manufacturer.

The warranty is limited to the repair or replacement of manufacturing defect parts and it doesn't include the repair of defects caused by:

- Damage or loss caused by improper use or abuse;
- Damage or loss caused by non-compliance of use and maintenance instructions.
- Damage or loss caused by acts of nature or catastrophes like fire, earthquakes, floods, lightning, etc.
- Damage or loss caused by a failure in compliance of the specified conditions for the system, such as improper installation, improper environment conditions, etc.
- Damage or loss caused by the use outside the territory where the system was originally sold;
- Damage or loss caused by Malware (malicious software such as virus, worm, among others that can damage computers);
- Fall or crash;

- inappropriate storage;
- Technical service performed by nonauthorized people by Alliage.

### ***Warranty period***

The manufacturer offers a warranty of **12 (twelve) months** starting from the installation date, against parts defects, raw materials, and manufacturing of its hardware products. In the case of defects occurred and notified during the warranty period, replacement parts and labor force will be supplied for the replacement or the items covered by the warranty.

**THE MANUFACTURER IS NOT RESPONSIBLE FOR ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUSIVE BUT NOT ONLY THE WARRANTIES IMPLIED FOR COMMERCIALIZATION OR SUITABILITY TO A PARTICULAR PURPOSE AND THE ONES RESULTING OF A TRANSACTION OR COMMERCIAL USE.**

The express warranty is not applicable to the following items: disposable parts, defect or damage caused by negligence, improper use, operator mistake, inappropriate installation, changings or non-specified operations of the product from the manufacturer hardware.

**PERFORMING THE CONDITIONS STATED ABOVE, THE MANUFACTURER WON'T BE RESPONSIBLE BY ANY DAMAGE, DIRECT OR INDIRECT, IRRELEVANT OR SIGNIFICANT, THAT CAN OCCUR AND REGARDLESS THE LEGAL THESIS PROPOSED, INCLUDING RESPONSIBILITY STRICT AND NEGLIGENCE.**

At no time the manufacturer will be responsible by any of the following circumstances:

- Performance, failure or interference from any net used by the Client;
- Performance, failure or interference caused by any equipment or third-party software;

## ELECTROMAGNETIC COMPATIBILITY

The **Intraoral Dental Digital Imaging Sensor** is intended for use in the electromagnetic environment specified below. Buyer or user should ensure that it is used in such an environment.

The **Intraoral Dental Digital Imaging Sensor** is suitable for use in a professional health care environment, not including areas where there are sensitive equipment or sources of intense electromagnetic disturbances, such as the RF shielded room of an imaging system magnetic resonance imaging, in operating rooms near active AF surgical equipment, electrophysiology laboratories, armored rooms or areas where short wave therapy equipment is used.

The following tables provide information on compliance of the equipment according to the standard EN 60601-1-2:2015.


### ***GUIDANCE AND DECLARATION FOR ELECTROMAGNETIC EMISSIONS***

<b>Emission Essays</b>	<b>Conformity</b>	<b>Electromagnetic environments - guidelines</b>
RF emissions CISPR 11	Group 1	The <b>Intraoral Dental Digital Imaging Sensor</b> uses RF energy only for its internal functions. Therefore, your RF emissions are very low and probably will not cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <b>Intraoral Dental Digital Imaging Sensor</b> is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuation/ Scintillation Emissions IEC 61000-3-3	Not applicable	

**GUIDANCE AND DECLARATION FOR ELECTROMAGNETIC IMMUNITY**

<b>Phenomenon</b>	<b>Basic standard of EMC</b>	<b>Immunity test level</b>	<b>Level of conformity</b>
Electrostatic discharge	IEC 61000-4-2	±8 KV contact ± 2 KV, ± 4 KV, ± 8 KV, ± 15 KV air	±8 KV contact ± 2 KV, ± 4 KV, ± 8 KV, ± 15 KV air
Radiated RF EM fields	IEC 61000-4-3	3 V/m 80 MHz – 2.7 GHz 80 % AM at 1 kHz	3 V/m 80 MHz – 2.7 GHz 80 % AM at 1 kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	See table	See table
Electrical Fast / Transients bursts	IEC 61000-4-4	± 1 kV 100 kHz repetition frequency	± 1 kV 100 kHz repetition frequency
Conducted disturbances induced by RF fields	IEC 61000-4-6	3 V 0.15 MHz – 80 MHz 6 V m in ISM bands between 0.15MHz e 80MHz 80 % AM at 1 kHz	3 V 0.15 MHz – 80 MHz 6 V m in ISM bands between 0.15MHz e 80MHz 80 % AM at 1 kHz
Rated power frequency magnetic fields	IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz

Proximity fields from wireless RF communications equipment						
Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity Test Level (V/m)
385	380-390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
450	430-470	GMRS 460,FRS 460	FM $\pm$ 5 kHz deviation 1kHz sine	2	0.3	28
710	704-787	LTE 13, 17 Band	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE 5 Band	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE 1, 3, 4, 25 Band, UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450, LTE 7 Band	Pulse modulation 217 Hz	2	0.3	28
5240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
5500						
5785						

	The <b>Intraoral Dental Digital Imaging Sensor</b> is intended for the acquisition of intra-oral medical images of teeth, mandible and oral structures, and it is for exclusive dental use only. In case of EM disturbances, the operator may experience loss of communication between the device and the computer.
---	---



The compliance with EMC and EMI regulations can't be guaranteed by the use of modified cables or those which doesn't comply with the same standards under what the equipment was validated.



This equipment must not be used adjacent or supported by other equipment. The recommendations of this manual must be followed.



Don't use accessories, transducers, internal parts of components and other cables other than those previously specified by the manufacturer. This may result in increased emission or decreased electromagnetic immunity and result in improper operation.



Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm to any part of the Intraoral Dental Digital Imaging Sensor, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.



To maintain basic safety in relation to electromagnetic disturbances during the expected service life, always use the equipment in the specified electromagnetic environment and follow the maintenance recommendations described in this manual.